seca

seca 210

Deutsch	Seite 3
English	Page 5
Français	Page 7
Español	Página 9
Italiano	Pagina 11
Dansk	Side 13
Svenska	Sida 15
Norsk	Side 17
Suomi	Sivu 19
Nederlands	Pagina 21
Português	Página 23
Ελληνικά	Σελίδα 25

Herzlichen Glückwunsch

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Hamburg

Congratulations!

In the seca 210 measuring mat you have acquired an accurate and at the same time sturdy piece of equipment for measuring the length of babies.

The seca 210 measuring mat is designed for use in paediatric practices, paediatric hospitals or for home visits by midwives and paediatricians.

The measuring mat can be rolled up and so requires only a little space. There are also holes in the head stop on the mat to enable it to be hung on a wall.

Safety

Before using the new measuring mat, please take a little time to read the following safety instructions.

Follow the operating instructions.

- Never leave a baby lying on the measuring mat unsupervised.
- Only measure length on surfaces from which the baby cannot fall.
- Ensure that the measuring mat is cleaned and disinfected regularly to avoid infectious diseases being passed on.
- Ideally, a second person is required to measure length exactly.

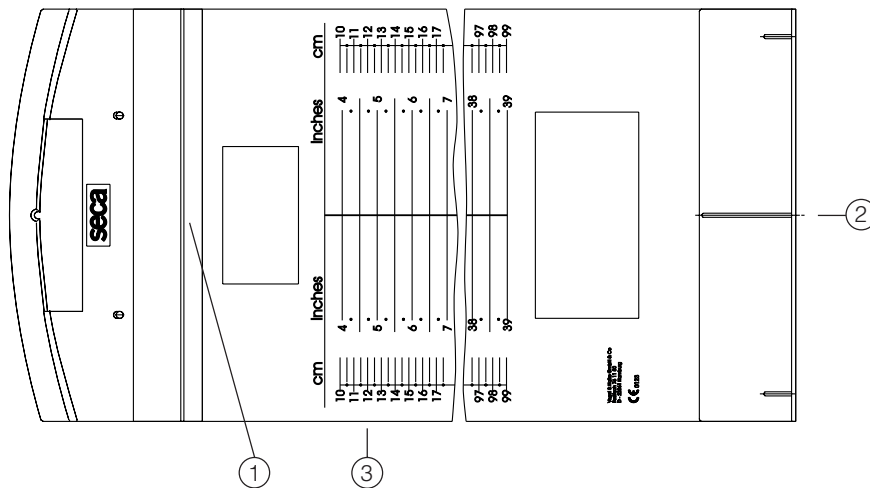
Operation

Measurement method:

1. Lie the baby on the measuring mat. The baby's head must lie against the head stop (1) of the measuring mat.
2. Using your right hand, push the foot stop (2) in the direction of the baby's feet. The left hand holds the baby in position.

3. Stretch the baby's legs by pressing its knees carefully and gently down to the mat with the left hand.
4. With your right hand, rest the foot stop (2) at right-angles to the soles of the baby's feet.
5. Read off body length at the marking on the measuring mat (3).

If the above steps are followed, accuracy greater than ± 5 mm is achieved.

Cleaning

Clean the measuring mat as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Technical data

Measurement range: 100 - 990 mm /
4 - 39 inches
Graduations: 5 mm / 1/4 inch
Dimensions
(W x H x D): 300 mm x 140 mm x
1340 mm
Weight: approx. 500 g
Temperature range: +10 °C to +40 °C
Medical product in
accordance with
Directive 93/42/EEC: Class I with measuring
function

Disposal

Should it no longer be possible to use the measuring mat, the responsible waste disposal association will be glad to give you information on how to dispose of it properly.

Guarantee period

We give a guarantee of **1 year** from the date of purchase for defects attributable to poor materials or workmanship. Defects which come under guarantee are repaired free of charge. Other claims cannot be taken into account. Transport to and from ourselves will be at the customer's expense if the equipment is located somewhere other than at the purchaser's place of residence. The guarantee is invalidated if the equipment is mishandled.

We would ask customers abroad to contact the sales organisation of the appropriate country in the event of a claim under guarantee.

Important: guarantee valid only in combination with proof of purchase!

CE declaration of conformity

Mechanical equipment for measuring length

seca model 210

Serial no.: see reverse of measuring mat

The measuring mat meets the applicable requirements of Directive 93/42/EEC concerning medical products.

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Vogel & Halke GmbH & Co