# User Manual

## Ambulance Stretcher YXH-2A

(EU) MDR 2017/745

Basic UDI-DI:6974580870001DN

### Accessories:

1.two safety belts 2.fixing device 3.IV pole 4.mattress

### I. Technical Data:

1.Product Size (L×W×H):190×55×50cm

2.Carton Size(1pcs): 195×62×34cm

3.Max angle of the back:75°

4.N.W.:25.5kg G.W.:31.5kg

5.Load Bearing: ≤ 159kg

### II. Description:

1.The stretcher surface is made of soft sponge mattress, which makes patient feel comfortable

2.Pressing the button under the head of stretcher, the head will be raised or lowered.

3.After the stretcher arrive destination, people can pull out the handle (for lifting the stretcher) & so take away the stretcher by two persons.

4.The fenders are of the return elastic locking .Turning up the fenders for patient's safety sake. If necessary, safety belt is to be used.

5.When infusion is needed for patient, IV stand can be inserted in one end of the stretcher.

### III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital, battlefield and gym to carry the wounded & patients. It also can be used in general operation.

### IV. Cautions:

1.When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.

2.In general operation, be careful not to out torn the stretcher surface.

### V. Maintenance:

1.Keep clean as routine (incl. sterilization).

2.Often check if any parts loose or not.

Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd

No.7th, Middle Xinzha Road, Zhashang Industrial Zone,

Yangshe Town, Zhangjiagang City, Jiangsu 215600, China

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Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd  
No.7th, Middle Xinzha Road, Zhashang Industrial Zone,  
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China  
SRN: CN-MF-000008449

**Importador:** Quirumed S.L.U. CIF B97267405 C/ Corretger 117, Pq. Emp. Tàctica, Paterna, Valencia (Spain)

SUNGO Europe B.V.  
Olympisch Stadion 24, 1076 DE  
Amsterdam, Netherlands  
SRN: NL-AR-000000247

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