User Manual

Ambulance Stretcher YXH-3A

(EU) MDR 2017/745

Basic UDI-DI :6974580870001DN

Accessories:

1.two safety belts 2.fixing device 3.IV pole 4.mattress

I. Technical Data:

1.High position: (L×W×H):190×55×92cm , Low position: (L×W×H):190×55×26cm

2.Carton Size(1pcs): 195×62×28cm

3.Max angle of the back:75°

4.N.W.:36kg G.W.:43kg

5.Load Bearing: ≤ 159kg

II. Description:

1.The stretcher surface is made of soft foamed cushion in order to make patient feel comfortable.

2.One person can send the patients to the ambulance car, When the patient need low position, pull out the safety handle & then press it, the stretcher' s height will automatically lower down.

3.When the destination arrived, pulling out the safety handle, the stretcher' s legs will unfold automatically

4.There are the return elastic lock device at both side of the stretcher & the leg parts. Lock tightly when turn up for patient safety.

5.Pressing the button under the stretcher, the head part will be raised or lowered.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital, battle field and gym to carry the wounded & patients. It also can be used in general operation.

IV. Cautions:

1.When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.

2.In general operation, be careful not to out torn the stretcher surface.

V. Maintenance:

1.Keep clean as routine (incl. sterilization).

2.Often check if any parts loose or not.

Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd

No.7th, Middle Xinzha Road, Zhashang Industrial Zone,

Yangshe Town, Zhangjiagang City, Jiangsu 215600, China

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Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd
No.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China
SRN: CN-MF-000008449

Importador: Quirumed S.L.U. CIF B97267405 C/ Corretger 117, Pq. Emp. Tàctica, Paterna, Valencia (Spain)

SUNGO Europe B.V.
Olympisch Stadion 24,1076 DE
Amsterdam, Netherlands
SRN: NL-AR-000000247

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