# User Manual

## Ambulance Stretcher YXH-3A5

(EU) MDR 2017/745

Basic UDI-DI :6974580870001DN

### Accessories:

1.two safety belts 2.fixing device 3.IV pole 4.mattress

### I. Technical Data:

1.High position: (L×W×H):190×55×92cm , Low position: (L×W×H):190×55×26cm

2.Carton Size(1pcs): 195×62×28cm

3.Max angle of the back:75°

4.N.W.:39kg G.W.:42kg

5.Load Bearing: ≤ 159kg

### II. Description:

1. The stretcher surface is made of soft sponge mattress and adjustable back which make patient feel comfortable. One person can operate.

2. When need low position, pull out the safety handle & then press it, the stretcher' s height will automatically lower down.

3. There are the return elastic lock device at both side of the stretcher & the leg parts. Lock tightly when turn up for patient safety.

4. Pressing the button under the stretcher, the head part will be raised or lowered.

5. This stretcher' s loading end is removable. It can be used in narrow area, like lift etc.

### III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital, battle field and gym to carry the wounded & patients. It also can be used in general operation.

### IV. Cautions:

1.When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.

2.In general operation, be careful not to out torn the stretcher surface.

### V. Maintenance:

1.Keep clean as routine (incl. sterilization).

2.Often check if any parts loose or not.

Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd

No.7th, Middle Xinzha Road, Zhashang Industrial Zone,

Yangshe Town, Zhangjiagang City, Jiangsu 215600, China

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SUNGO Europe B.V.