User Manual

Ambulance Stretcher YXH-3F

(EU) MDR 2017/745

Basic UDI-DI :6974580870001DN

Accessories:

1.two safety belts 2.fixing device 3.mattress

I. Technical Data:

1.High position: (L×W×H):198×55×87cm , Low position: (L×W×H):198×55×28cm

2.Carton Size(1pcs): 200×63×35cm

3..N.W.:26kg G.W.:35kg

4.Load Bearing: ≤ 159kg

II. Description:

1. The stretcher surface is made of soft sponge mattress which makes patient feel comfortable.

2.When adjusting the stretcher to a low position, should let Two Person pull out the handles at the both ends of the stretcher simultaneously.

3.When getting onto the ambulance car, only one person can control the stretcher. One person must stand on the left of the stretcher, and then pull the handle controller, the right legs of the stretcher will bend down, and it will get onto the stretcher.

4.After reached destination, pulling out the safety handle, the stretcher' s legs will unfold automatically.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital, battle field and gym to carry the wounded & patients. It also can be used in general operation.

IV. Cautions:

1.When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.

2.In general operation, be careful not to out torn the stretcher surface.

V. Maintenance:

1.Keep clean as routine (incl. sterilization).

2.Often check if any parts loose or not.

Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd

No.7th, Middle Xinzha Road, Zhashang Industrial Zone,

Yangshe Town, Zhangjiagang City, Jiangsu 215600, China

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Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd
No. 7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China
SRN: CN-MF-000008449

Importador: Quirumed S.L.U. CIF B97267405 C/ Corretger 117, Pq. Emp. Tàctica, Paterna, Valencia (Spain)

SUNGO Europe B.V.
Olympisch Stadion 24, 1076 DE
Amsterdam, Netherlands
SRN: NL-AR-000000247

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Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd
No.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China
SRN: CN-MF-000008449

Importateur: Quirumed S.L.U. CIF B97267405 C/ Corretger 117, Pq. Emp. Tàctica, Paterna, Valencia (Spain)

SUNGO Europe B.V.
Olympisch Stadion 24, 1076 DE Amsterdam, Netherlands
SRN: NL-AR-000000247

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