User Manual

Scoop Stretcher YXH-4A

(EU) MDR 2017/745

Basic UDI-DI :6974580870001DN

Accessories:

1. two safety belts

I. Technical data:

1. Product size(L×W ×H):168 ×44 ×7cm, The longest can be up to 210cm.
2. Folding size(L×W ×H):120 ×44 ×9cm
3. Carton size(1pc):170 ×45 ×7cm
4. N.W.(1pc): 7.4kg G.W.:10kg
5. Load bearing: ≤159kg

II. Description:

- 1.It is a separately-type stretcher to transfer fractured and serious patients.
- 2.At both ends of the stretcher, a hinged and joined arrangement is set in the middle, which can set the stretcher into two parts and it is easy to lock and unlock.
- 3.The stretcher can strengthen the patients at original place; it can reduce the further hurt of the patients, the patients can be quickly & easily put in the stretcher and also the stretcher can be taken out from the back of the patient without moving the patient.
- 4.The length of the stretcher can be adjusted according to patients' body. The foot end is designed as a narrow-frame structure.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital, battlefield and gym to carry the wounded & patients.

IV. Attention:

When carrying patients with this stretcher, be sure fasten the safety belts for safety sake.

V. Maintenance:

- 1.Keep clean as routine (incl. sterilization).
- 2.Often check if any parts loose or not.

Zhangjiagang Xiehe Medical Apparatus & Instruments Co., Ltd
No.7th, Middle Xinzha Road, Zhashang Industrial Zone,
Yangshe Town, Zhangjiagang City, Jiangsu 215600, China
SRN: CN-MF-000008449

Importer: Quirumed S.L.U. CIF B97267405 C/ Corretger 117, Pq. Emp. Tàctica, Paterna, Valencia (Spain)

SUNGO Europe B.V.
Olympisch Stadion 24,1076 DE
Amsterdam, Netherlands
SRN: NL-AR-000000247

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