




User manual

A045-1 **Stainless Steel Gynecological Table**

Registration code: SKF-A

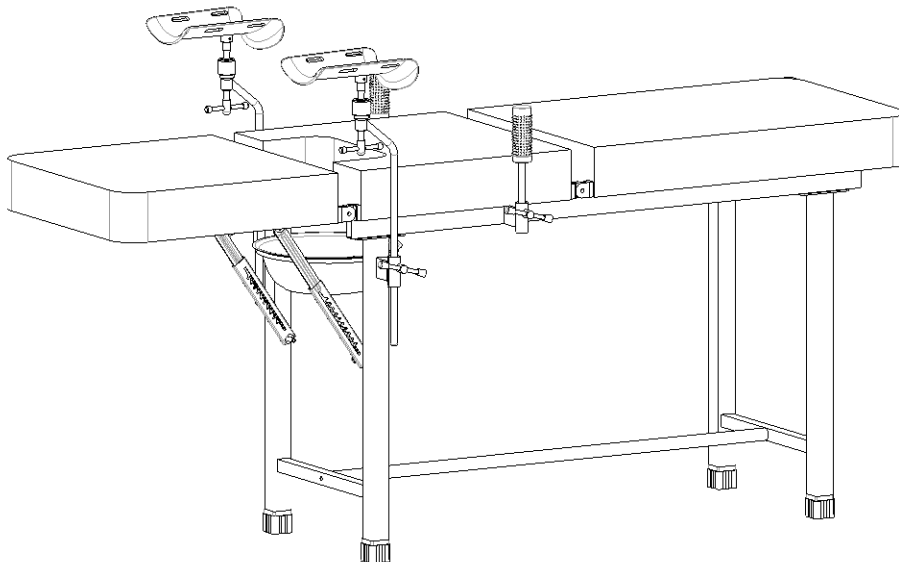

SAIKANG / Medical

Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098

Contents

Manual introduction	2
1. Installing guide	3
1.1 List of standard accessories	3
1.2 Bed foot installation	6
1.3 Connection pipe installation.....	7
1.4 Mechanical Level installation	8
1.5 Foot pedal installation	9
1.6 Hand bar installation.....	10
2. Use method	11
2.1 Back Section Adjustment.....	12
2.2 Leg Section Adjustment.....	13
3. Stainless steel gynecological table-maintenance.....	13
4. Packaging, transport, storage.....	14
5. After-sales service	14

Manual introduction

Note:

Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.





- ▶ This manual is applicable to A045-1 stainless steel gynecological table. It contains the installation method, operating instructions and maintenance check points.
- ▶ The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- ▶ This user manual includes the description, instruction, maintenance and technical parameters of the product.
- ▶ Any questions, please contact the after-sale service center or distributor.
- ▶ Anyone using or operating this product should read and comply with it.
- ▶ Except for some unexpected situations, users are required to be familiar with the usage and precautions.

1. Installing guide

1.1 List of standard accessories

After opening the packing case, please check whether the following parts are complete in the packing case:

Table a:

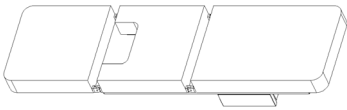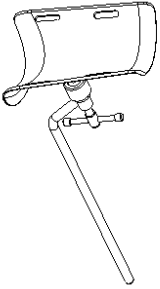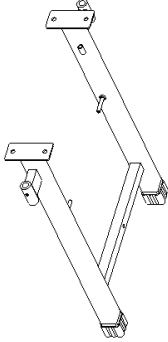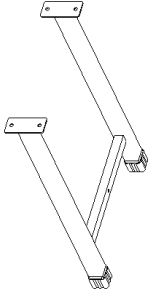NO.	Name	Picture	Specification	QTY	Unit
1	Bed plate		/	1	pc
2	Stirrup		/	2	pcs
2	Behind bed foot		L427*W653mm	1	pc
3	Front bed foot		L427*W653mm	1	pc









4	Basin		φ300*85mm	1	pc
5	Basin hoop		φ285-L80mm	1	pc
7	Connection pipe		Φ25*993-M8mm	1	pcs
8	Screw	/	Refer to Table a.1	1	set

Table a.1



NO.	Name	Picture	Specification	QTY	Unit
1	Hexagon screw		M10*25mm	8	pcs
2	Flat pad		Φ10*Φ21*H1.8mm	8	pcs
3	Spring washer		Φ10*Φ17*H3mm	8	pcs
4	Nut		M10mm	8	pcs
5	Hexagon screw		M8*40mm	2	pcs
6	Flat pad		Φ8*Φ16*H1.2mm	2	pcs
7	Spring washer		Φ8*Φ14*H2.5mm	2	pcs
8	Lock nut		M8mm	2	pcs
9	Gynecological bed full set		Seven pieces one set	2	pcs
10	Hand structure		Φ32mm	2	pcs
11	Plastic handle sleeve		Φ32mm	2	pcs

1.2 Bed foot installation



Lay a protective pad and put the instrument trolley body upside down;

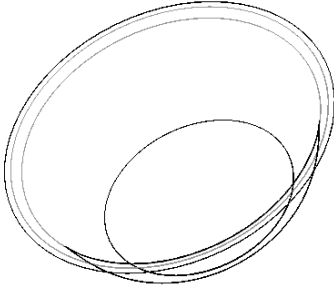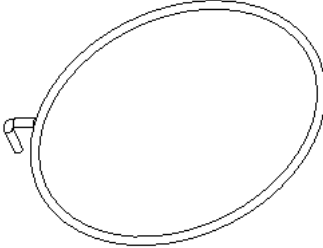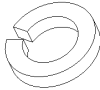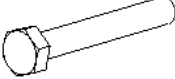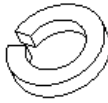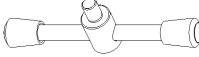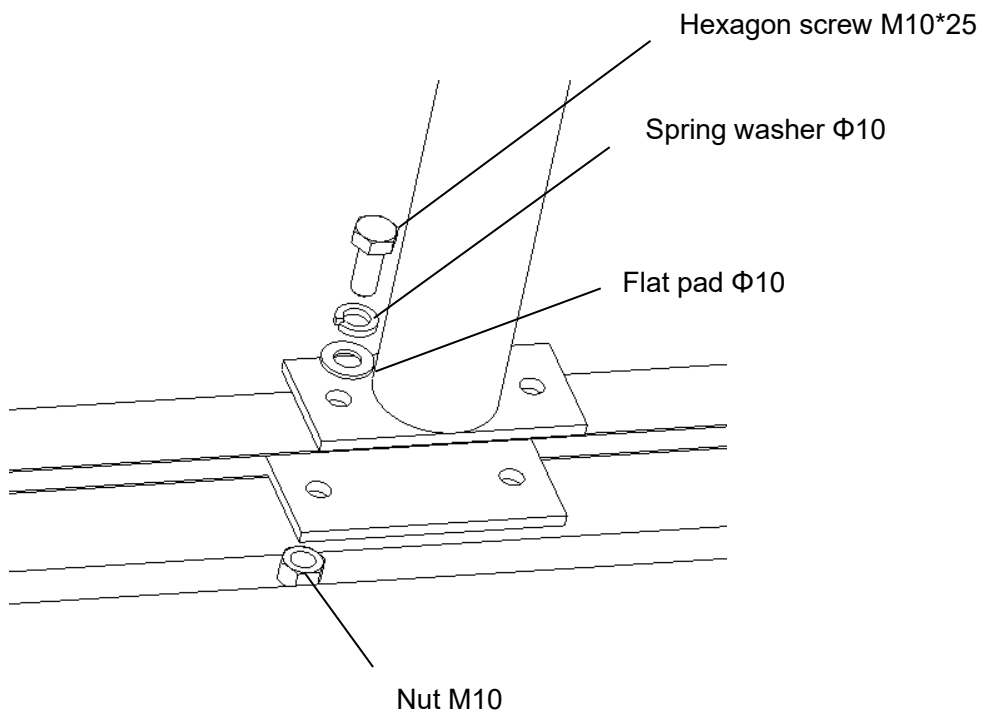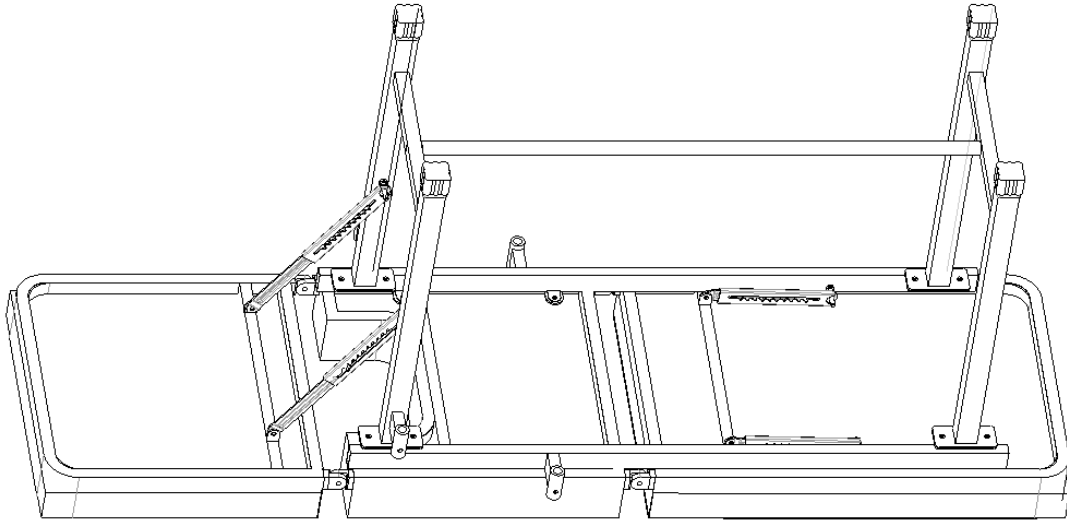

1.3 Connection pipe installation

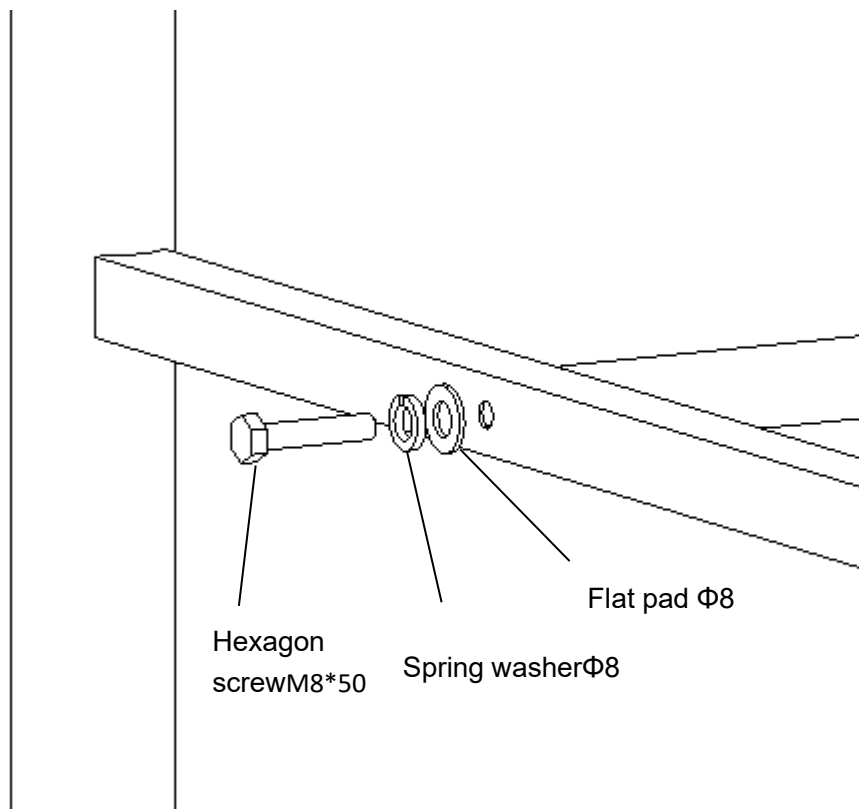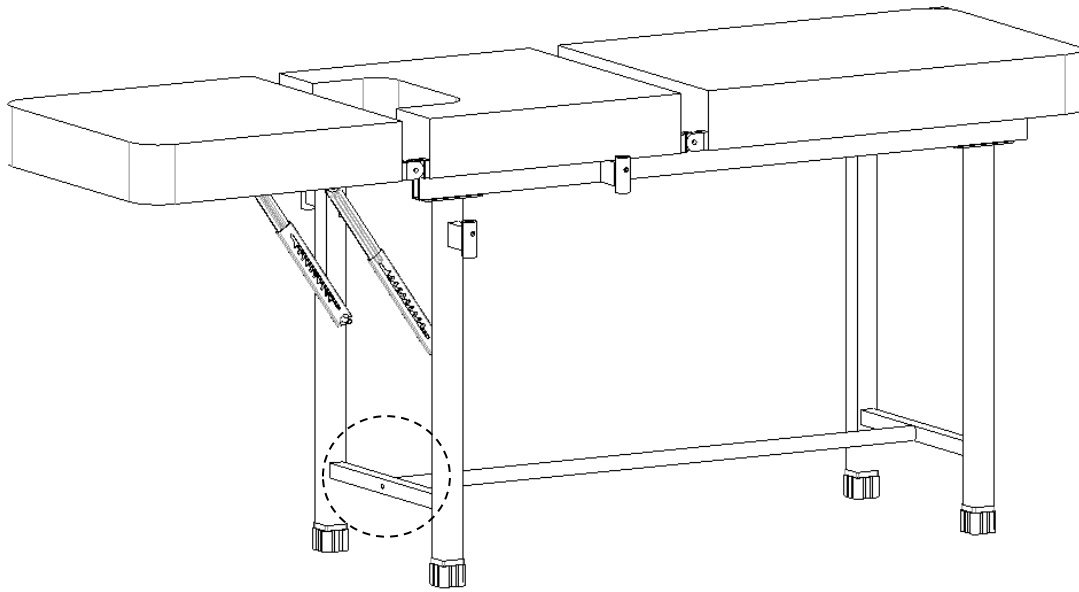


1.4 Mechanical Level installation

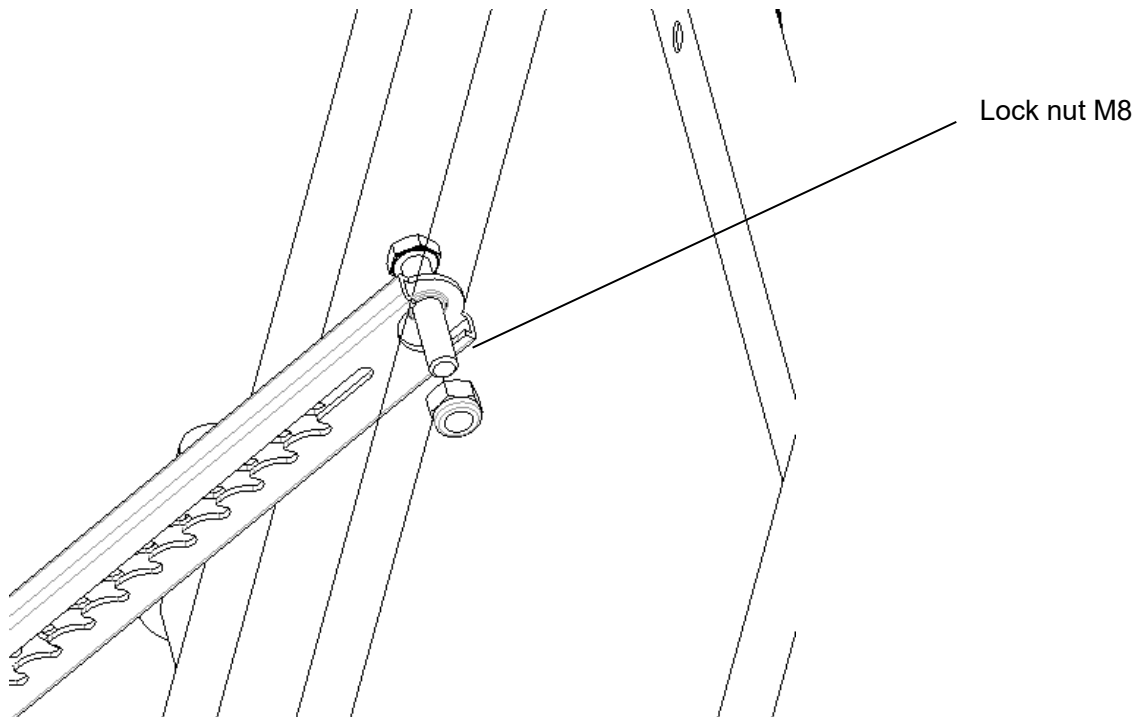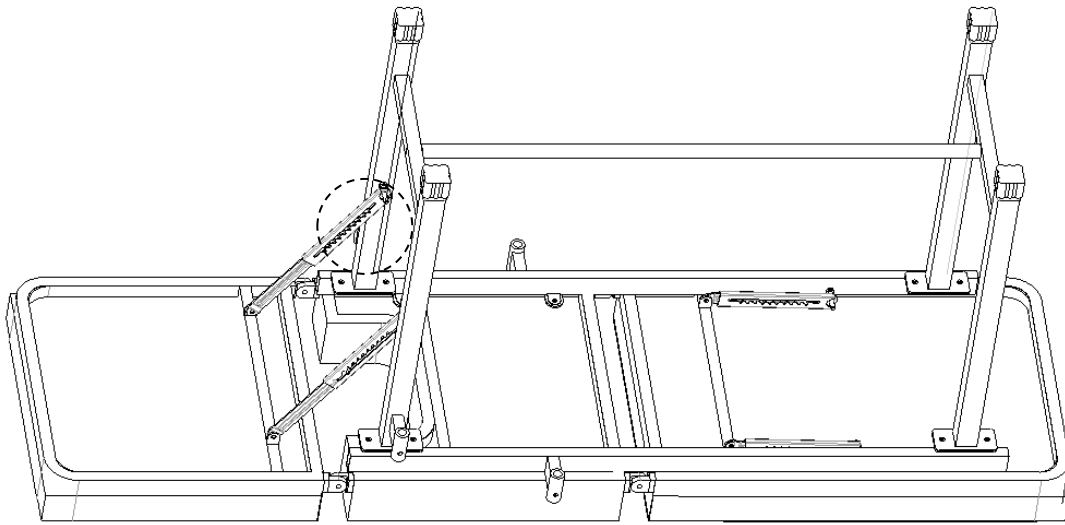


1.5 Foot pedal installation

Insert the pedal into the screw sleeve, turn the wrench 1 to adjust the height of the pedal, and turn the wrench 2 to adjust the angle of the pedal by 360°

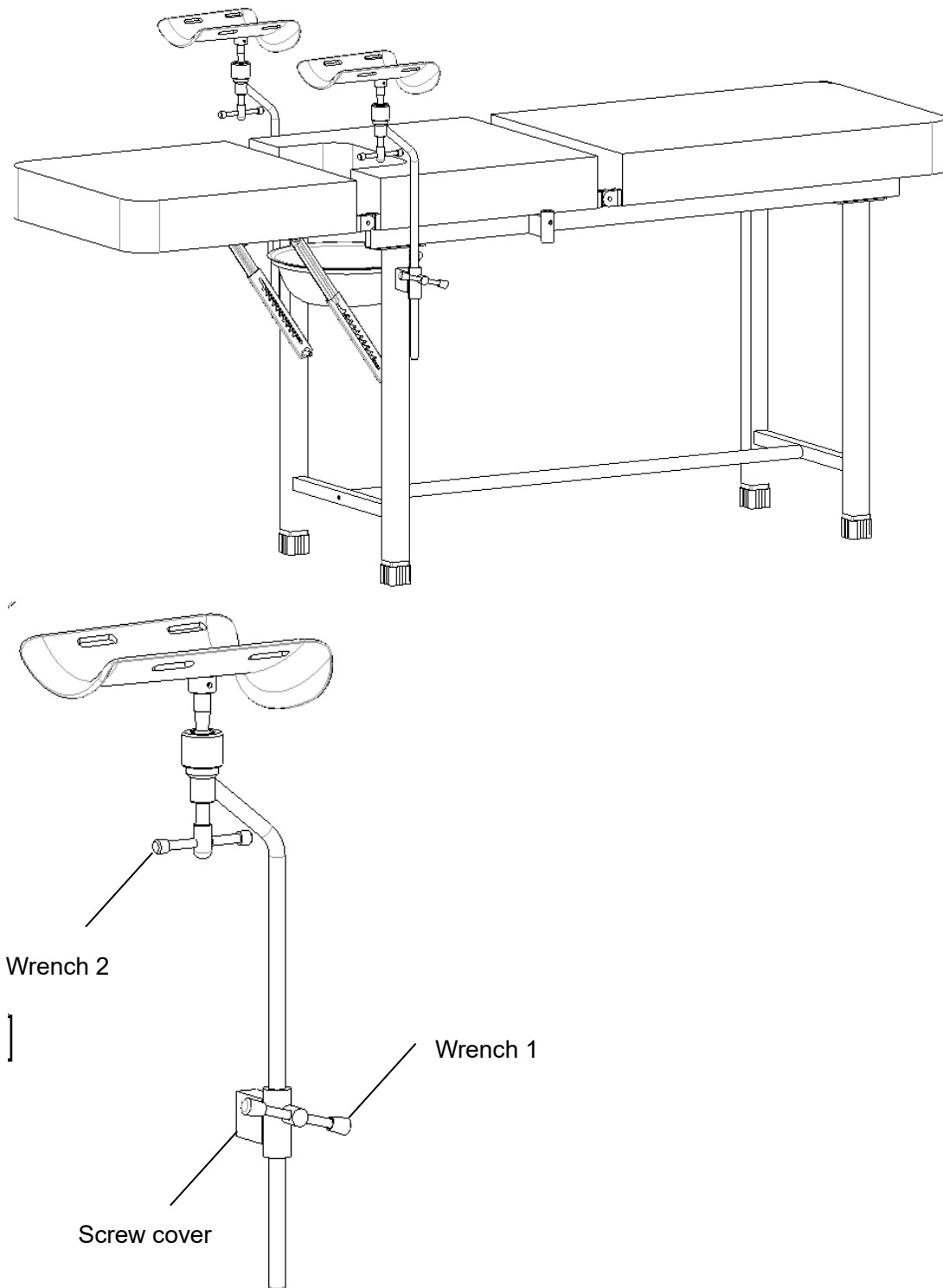


1.6 Hand bar installation

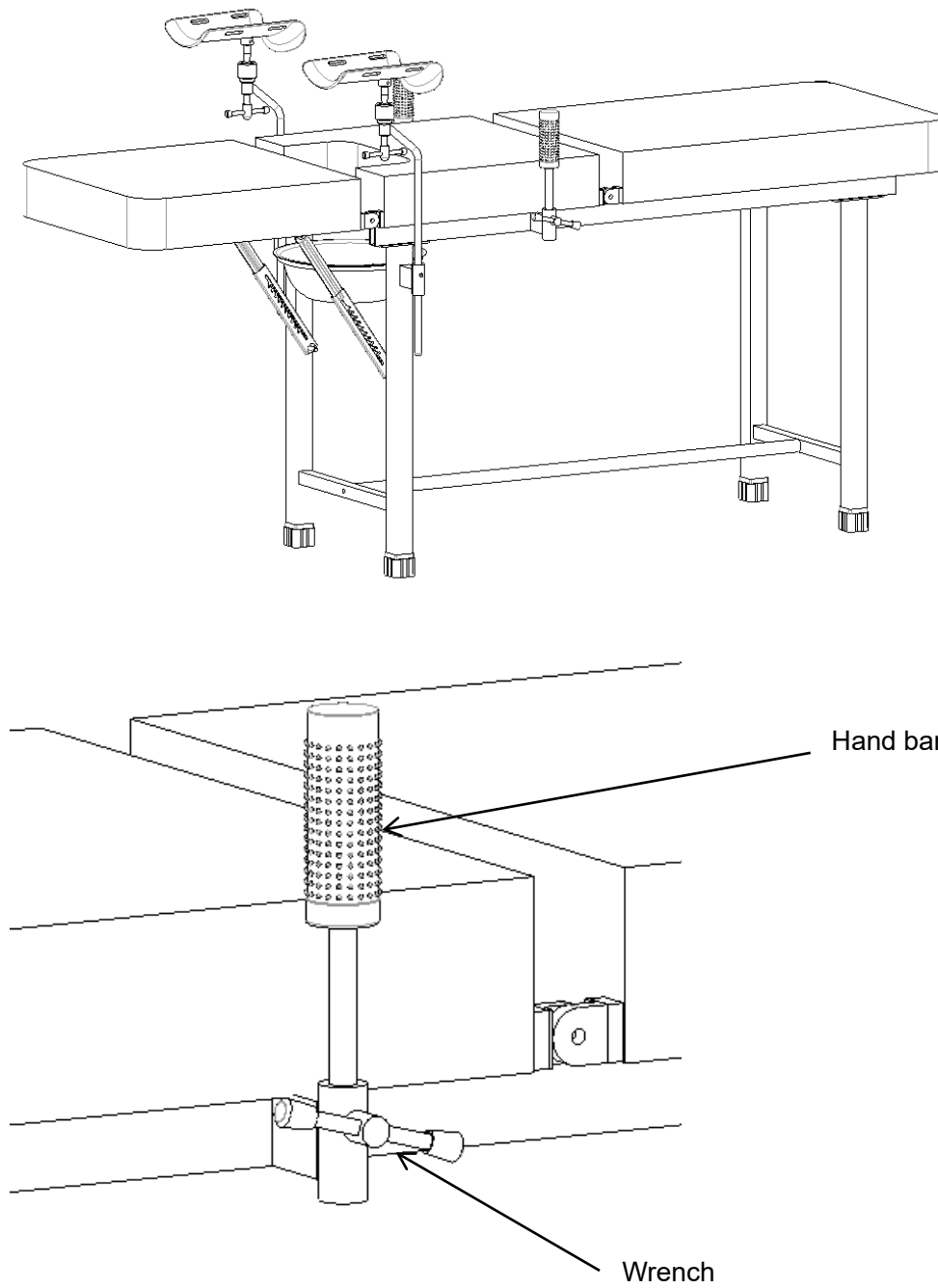

Turn the wrench can adjust the hand bar height


2.Use method

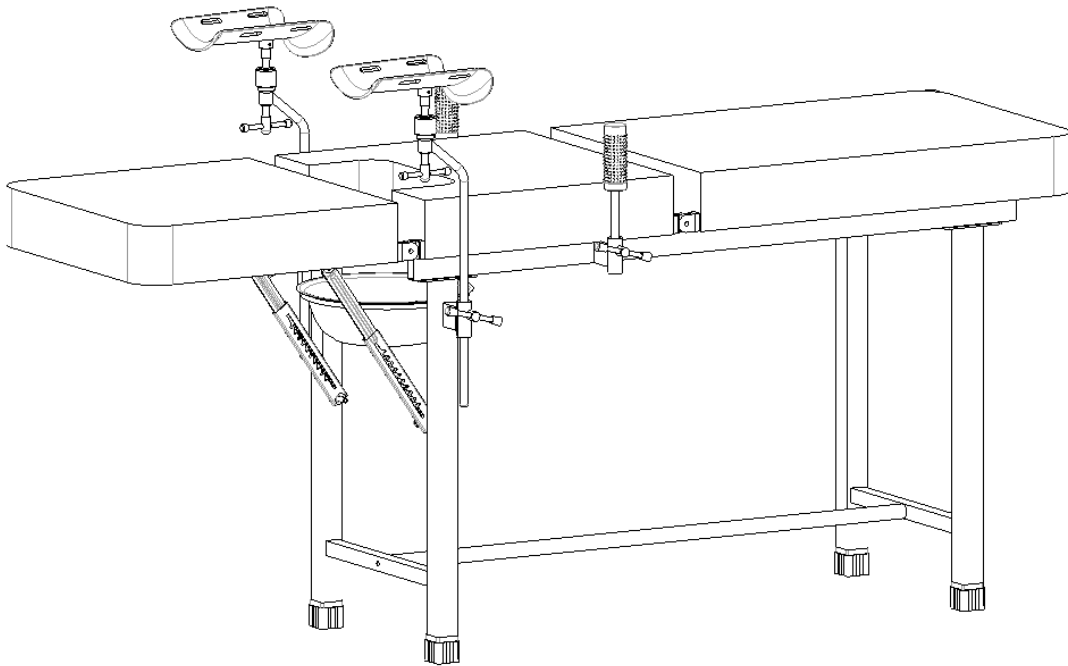

Dimension	Length	mm	1820±10
	Width		590±10
	Height		800±10
Weight	Safe load bearing	Kg	170


2.1 Back Section Adjustment

Lift the backplane directly to the desired angle to complete the backplane lifting. Raise the backplane to the maximum angle, and then lower the backplane to complete the backplane descent.

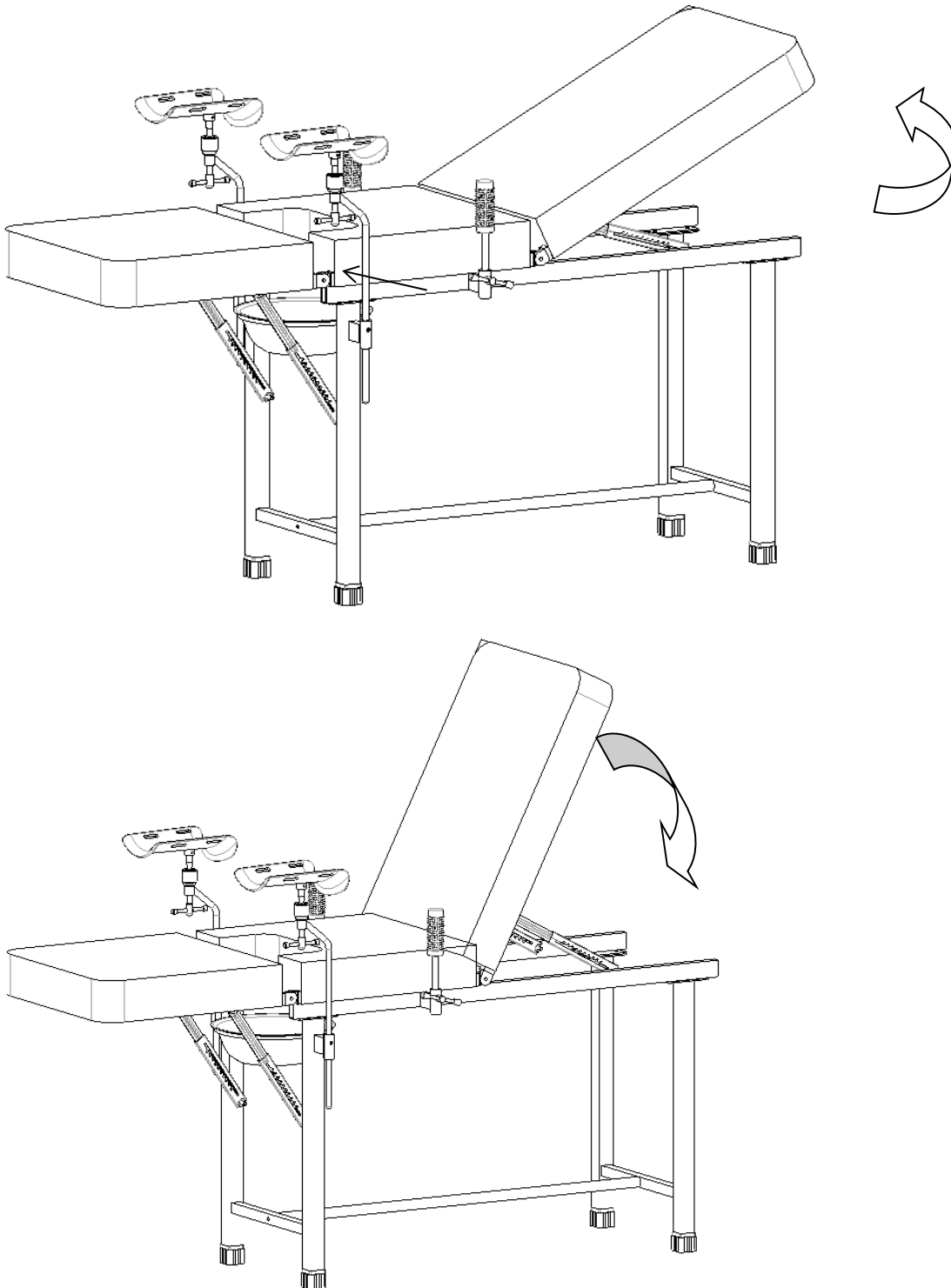

2.2 Leg Section Adjustment

Lift the leg plate directly to the maximum angle, and then slowly lower the back plate to complete the leg descent.

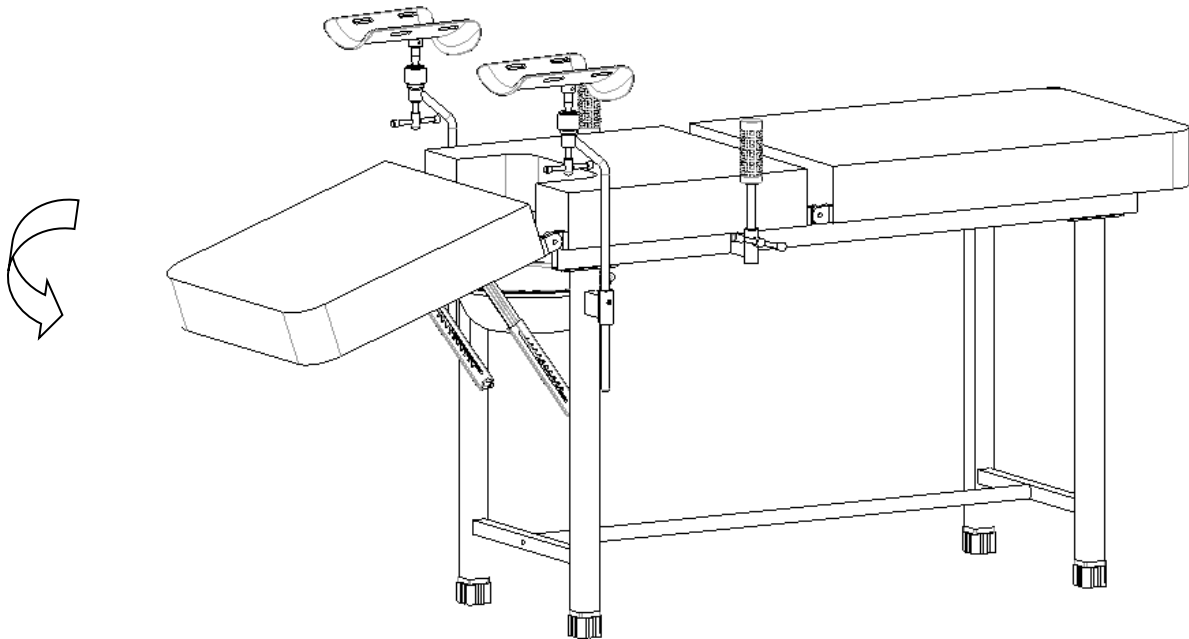

3. Stainless steel gynecological table-maintenance

1. In order to use the stainless steel gynecological table safely, safety inspections must be done regularly. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
2. When the stainless steel gynecological table ages and it reaches a certain servicing time, which the metal part of the treatment trolley and plastic parts of the stainless steel basin frame can be recycle.
3. Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
4. If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the treatment trolley, which will cause the stainless steel surface to rust.

4. Packaging, transport, storage

1. The packaging of stainless steel gynecological table is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and protection from sunlight and rain during the transportation of stainless steel gynecological table.
3. The stainless steel gynecological table should be stored:
 - a) Ambient temperature: $-40^{\circ}\text{C} \sim 70^{\circ}\text{C}$.
 - b) Relative humidity: $\leq 95\%$.
 - c) Atmospheric pressure: $500\text{hPa} \sim 1060\text{ hPa}$.

5. After-sales service

1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. Do not disassemble the internal device of this product by yourself to avoid unnecessary damage. If you find that the use is affected due to quality problems, please contact our after-sales service department or our authorized dealer.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD

GYNECOLOGICAL TABLE**Jiangsu Saikang Medical Equipment Co.,Ltd**

No. 35 Lehong Road, Modern Agriculture
Demonstration Park, Zhangjiagang City,
Jiangsu Province, China

**SUNGO Europe B.V.**

Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel,
The Netherlands



Importador: Quirumed S.L.U. CIF B97267405
C/Corretger 117 Pq.Emp.Tactica,Paterna,
Valencia(Spain)

LOT:HW25xxx



Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098



No. 35, Lehong Road · Zhangjiagang (Jiangsu) · China

+86 18021231901

sales@chinasaikang.com

www.saikangmedical.com