




User manual

SK-B1 **Stainless Steel Examination Table**

Registration code: SK-B1

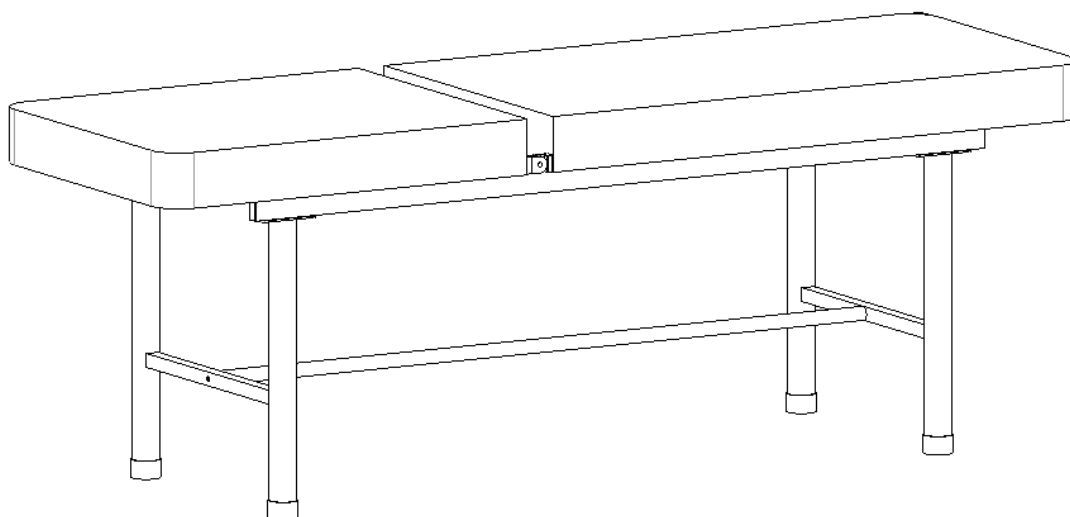

SAIKANG/Medical

Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098

Contents

Manual introduction	2
1. Installing guide	3
1.1 List of standard accessories	3
1.2 Foot structure installation.....	5
1.3 Connection pipe installation	6
2. Use method.....	7
2.1 Backrest lifting function	7
3. Stainless steel examination table-maintenance	8
4. Packaging, transport, storage.....	9
5. After-sales service	9

Manual introduction

Note:

Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.

- ▶ This manual is applicable to SK-B1 stainless steel examination table. It contains the installation method, operating instructions and maintenance check points.
- ▶ The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- ▶ This user manual includes the description, instruction, maintenance and technical parameters of the product.
- ▶ Any questions, please contact the after-sale service center or distributor.
- ▶ Anyone using or operating this product should read and comply with it.
- ▶ Except for some unexpected situations, users are required to be familiar with the usage and precautions.

1. Installing guide

1.1 List of standard accessories

After opening the packing case, please check whether the following parts are complete in the packing case.

Table a:










No.	Name	Picture	Specification	QTY	Unit
1	Couch plate		/	1	pc
2	Foot structure		539*543mm	2	pcs
3	Connecting pipe		Φ25*1.2*1253-M8mm	1	pc
4	Screw	/	Refer to Table a.1	1	set

Table a.1:



No.	Name	Picture	Specification	QTY	Unit
1	Outer hexagon screw		M10*25mm	8	pcs
2	Flat washer		Φ10*Φ21*H1.8mm	8	pcs
3	Spring washer		Φ10*Φ17*H3mm	8	pcs
4	Lock nut		M10mm	8	pcs
5	Outer hexagon screw		M8*40mm	2	pcs
6	Flat washer		Φ8*Φ16*H1.2mm	2	pcs
7	Spring washer		Φ8*Φ14*H2.5mm	2	pcs

1.2 Foot structure installation


Lay a protective pad and put the main structure upside down;

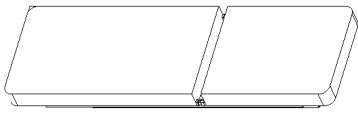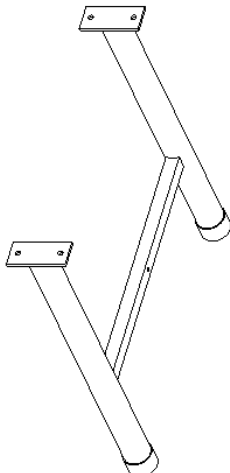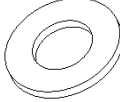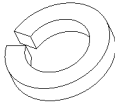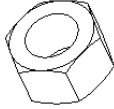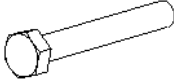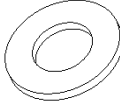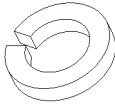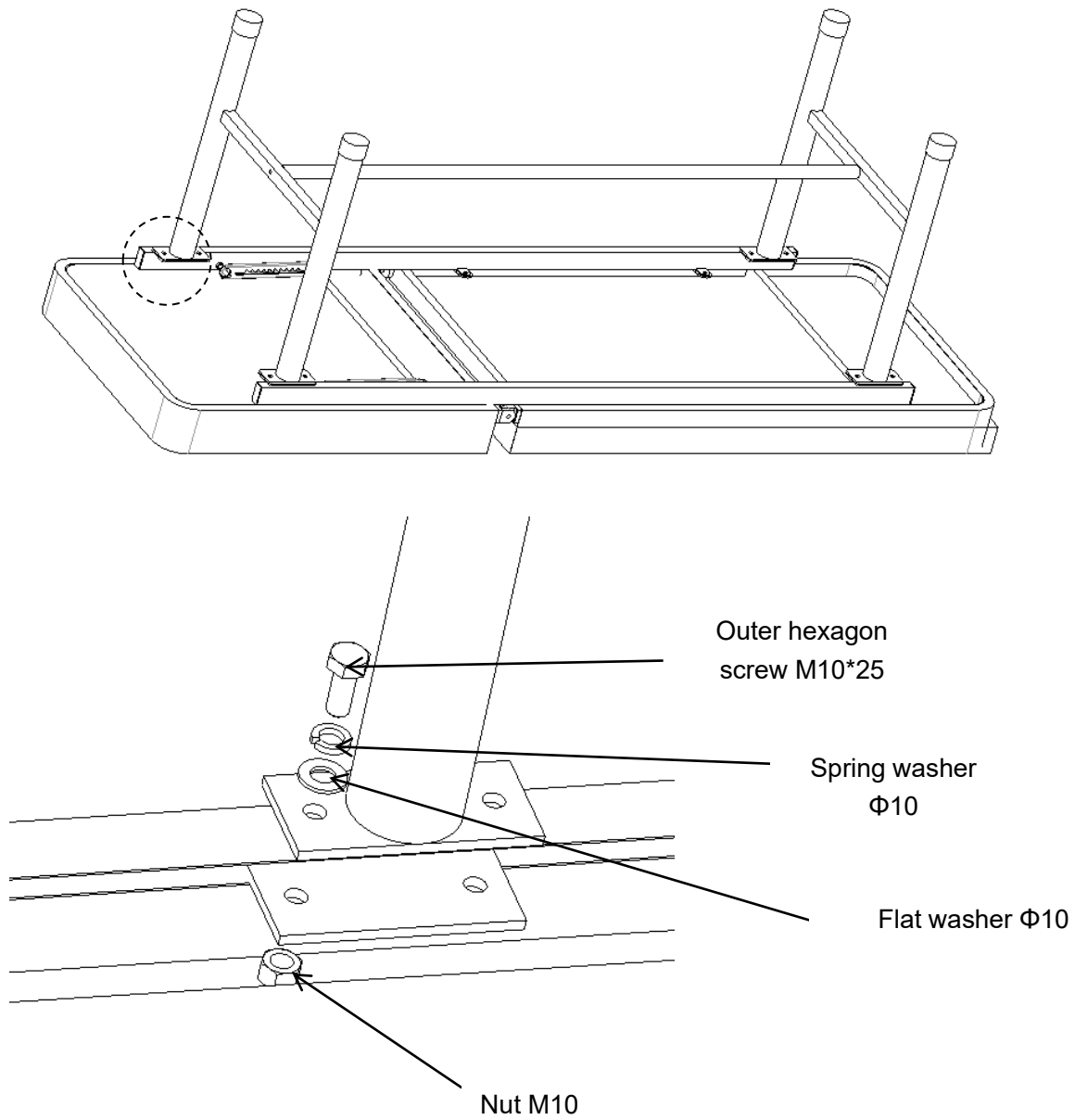

1.3 Connection pipe installation

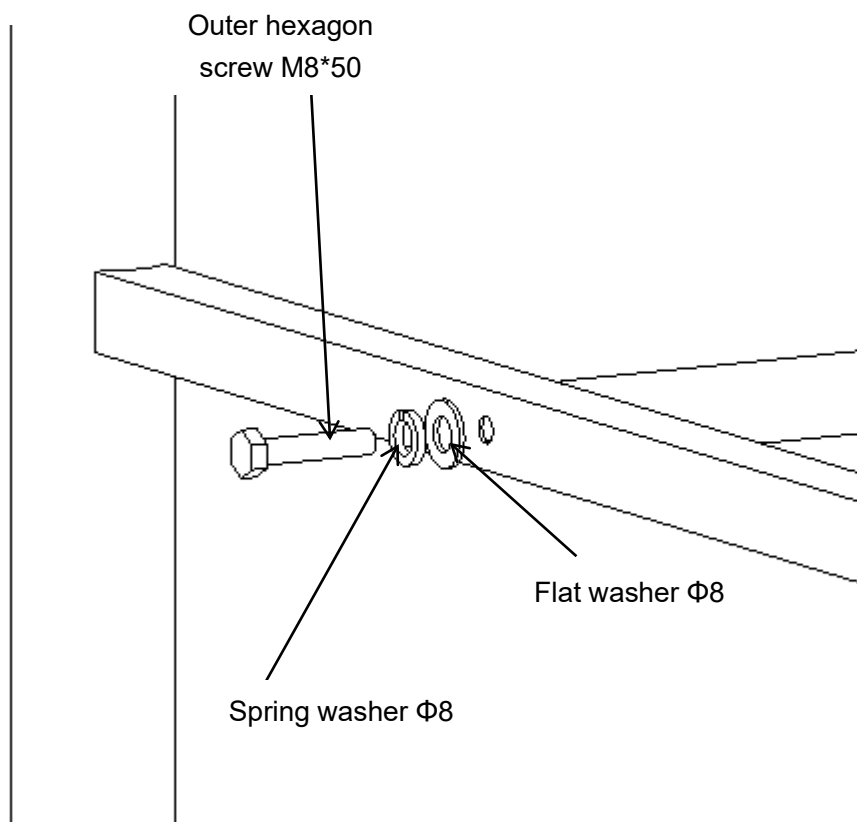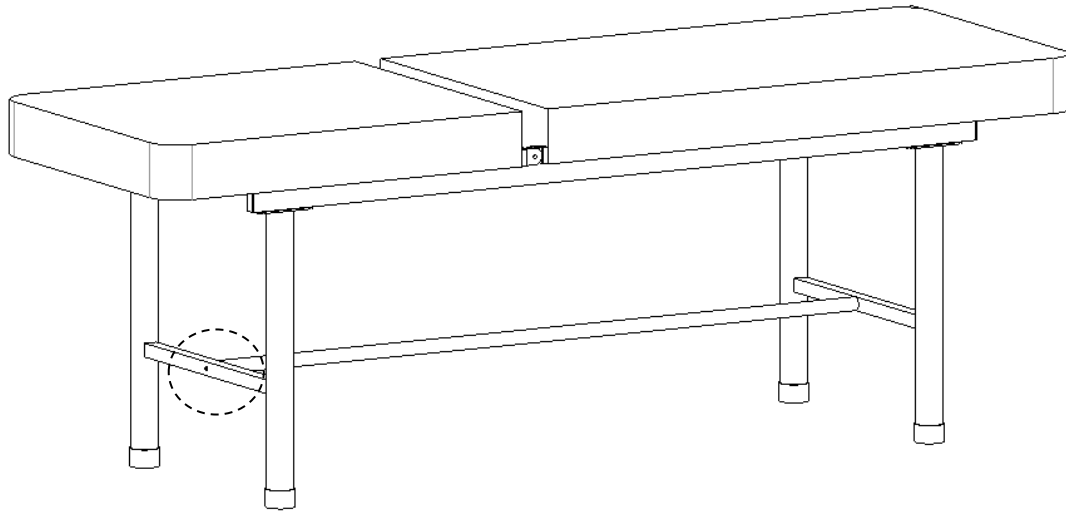


2. Use method

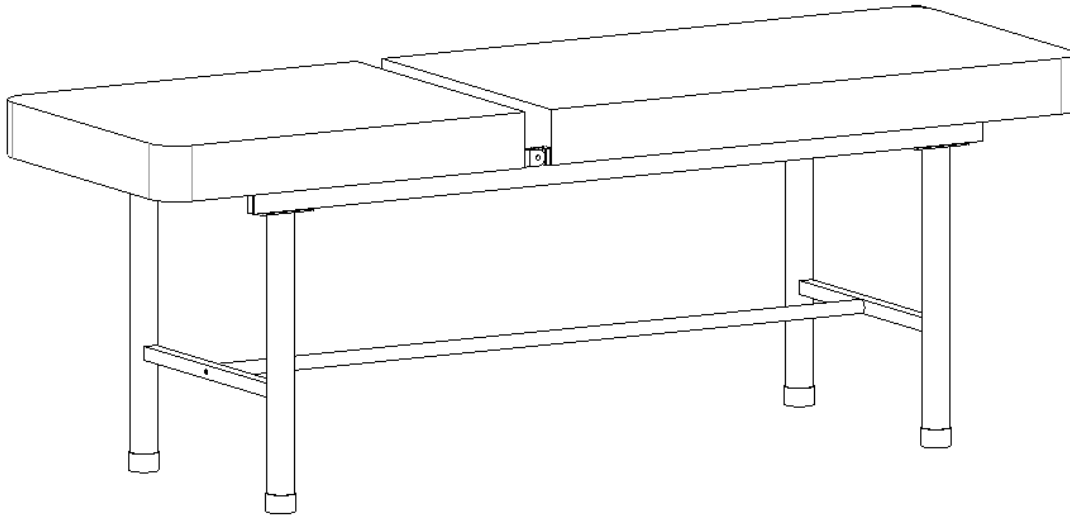


Dimensions	Length	mm	1900±10
	Width		600±10
	Height		680±10
Weight	Safe working load	Kg	170

2.1 Backrest lifting function

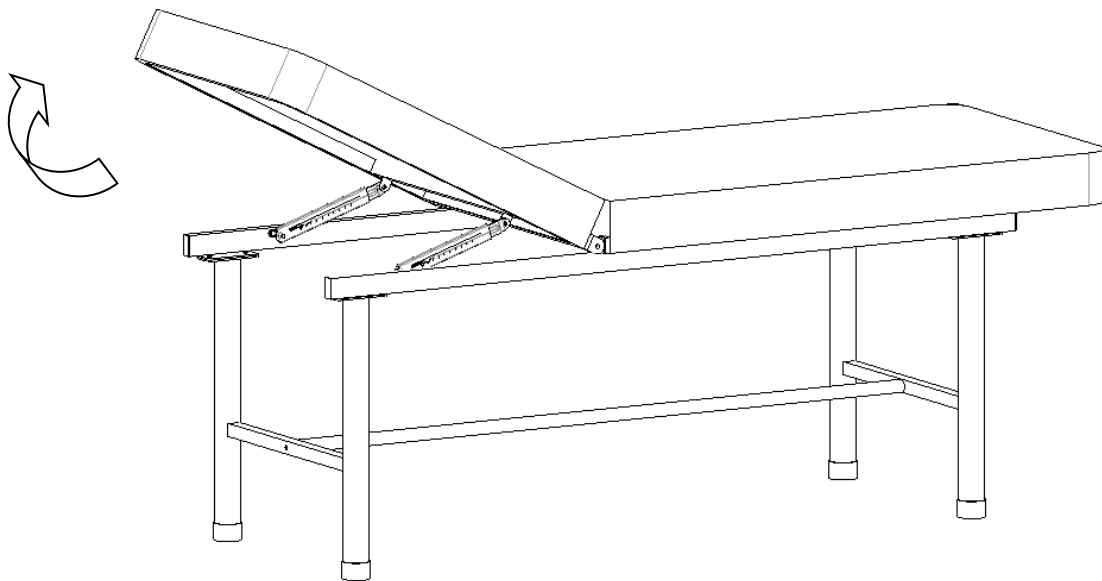

Lift the back plate to achieve the backrest lifting function, And lift the plate to the highest position,

then can put down the back plate.

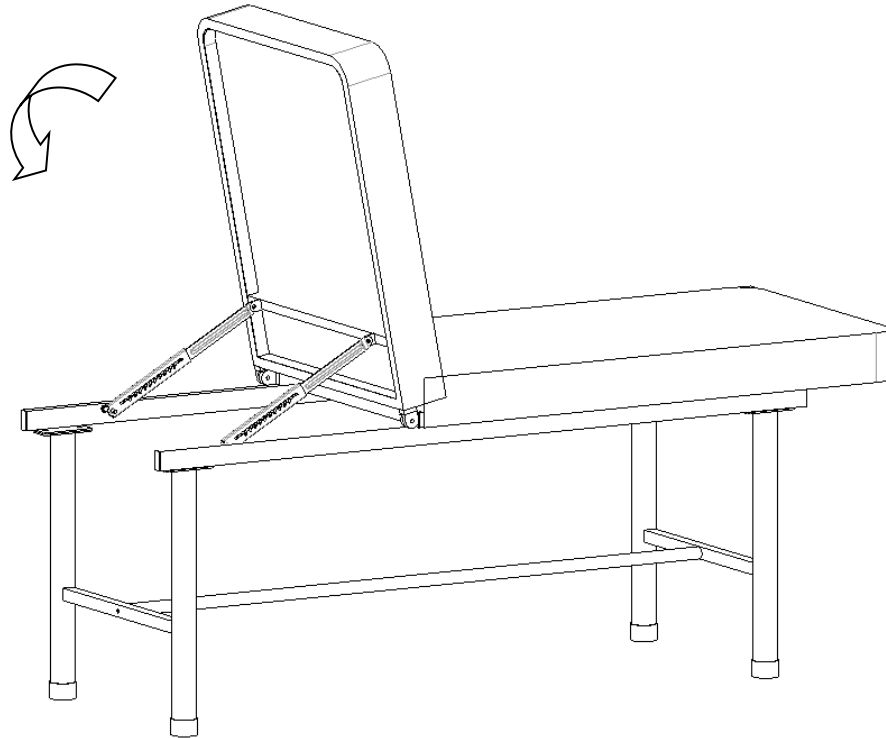

3. Stainless steel examination table-maintenance

1. In order to use the examination table safely, it is necessary to perform regular safety inspections on the examination table. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
2. When the examination table frame ages and it reaches a certain servicing time, which the metal parts and plastic parts of the examination table can be recycle.
3. Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
4. If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the examination table, which will cause the stainless steel surface to rust.

4. Packaging, transport, storage

1. The packaging of examination table is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and protection from sunlight and rain during the transportation of the examination table.
3. The examination table should be stored:
 - a) Ambient temperature: $-40^{\circ}\text{C} \sim +70^{\circ}\text{C}$.
 - b) Relative humidity: $\leq 95\%$.
 - c) Atmospheric pressure: $500\text{hPa} \sim 1060\text{hPa}$.

5. After-sales service

1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification, the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD



MODEL: SK-B1 | REF.: 319-SK-B1

MESA DE EXAMEN / EXAMINATION TABLE / TABLE D'EXAMEN / TAVOLO DA VISITA / MESA DE EXAME / Untersuchungstisch



Jiangsu Saikang Medical Equipment Co.,Ltd

No. 35 Lehong Road, Modern Agriculture
Demonstration Park, Zhangjiagang City,
Jiangsu Province, China



SUNGO Europe B.V.

Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel,
The Netherlands



Quirumed S.L.U. CIF B97267405

C/Corretger 117 Pq.Emp.Tactica,Paterna,
46980, Valencia(Spain)

LOT: HW26xxx



2026-xx-xx



Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098



No. 35, Lehong Road · Zhangjiagang (Jiangsu) · China

+86 18021231901

sales@chinasaikang.com

www.saikangmedical.com