

Manual Hospital Bed

Model: SK033-1

User's Manual



Picture for reference only



Contents

1. Manual Introduction.....	2
2. Applicable Scope.....	2
3. Features.....	2
4. Technical Parameters.....	2
5. Install Guide.....	2
6. Use Method.....	7
7. Notes.....	10
8. Warning and Caution.....	10
9. Maintenance.....	11
10. Storage.....	11
11. Transportation.....	11
12. List of Products.....	11

Note: The appearance of products described in this manual may be some difference from the finished products. This will not affect products performance. Please understand if it causes you any inconvenience.

1. Manual Introduction

This manual is a part of the article to which it is referred. Please read the instructions carefully as they provide important information on SAFETY DURING USE AND MAINTENANCE PROCEDURES descriptions and images contained in the publication are not binding; we have the right to modify any information for enhancement purposes without any obligation to update this manual. Graphics and images in this manual are just examples shown to you. Any improvement of products is available.

2. Applicable Scope

It is mainly used for hospital, nursing home, home care.

3. Features

1. The bed surface is powder coated, with punching holes.
2. The ABS bed head&foot can be easily assembled and disassembled.
3. The guardrail can be taken upwards, also downwards.
4. The angle of the back and the knee part can be adjusted by manual handles which makes the patient feel comfortable.
5. It characterized by its being safely-used, easy to disinfect and clean.

4. Technical parameters

Model No.	A100s
Overall size	0°-75°(±5°)
Max loading	250kg
Back-rest lifting angle	2100mm*970mm*500mm

5. Install guide

For saving the packing volume and safe transportation, the hospital bed you received is not installed. You may need to install the head and foot board, casters, turning table (if you chose).

Install Guide comes together with the User's Manual. Please carefully read this guide and strictly follow the steps.

Note: Make sure the bed installed completely before putting it into use.

Parts need to be prepared before installation:

The main tools and screw			
 1	 2	1.Drainage of hook 2.Nut	*2pcs
 1		1. screw	*10pcs

 1 2 3 4 5	1.Screw 2.Spacer 3.Spacing shim 4.Nut 5.Spring washer	*4pcs *4pcs
 1	1.Bed leg	*2pcs
 1 2 3 4	1.Short screw 2.Spring washer 3.Spacing shim 4.Nut	*8pcs
 1 2	1. Bolts and nuts 2.Spacing wedge	*2pcs
 1 2 3	1.Spacing shim 2.Nut 3.Spring washer	*2pcs
 1	 2	1.Bigger spanner: 8mm screw: 14 10mm screw:17 2.Small spanner: 8mm lock nut:13 10mm lock nut:17 *1pcs
 1	Screwdriver	*1pcs


Put all screws and nuts to right place, then tight together!!

Step1: The bed flat . Figure 1 shows (1) :





Step 2: Install drainage hook. Figure 2 shows:

Install the drainage hook by twisting it to the bed, Are mounted on both sides of the bed beams bed.

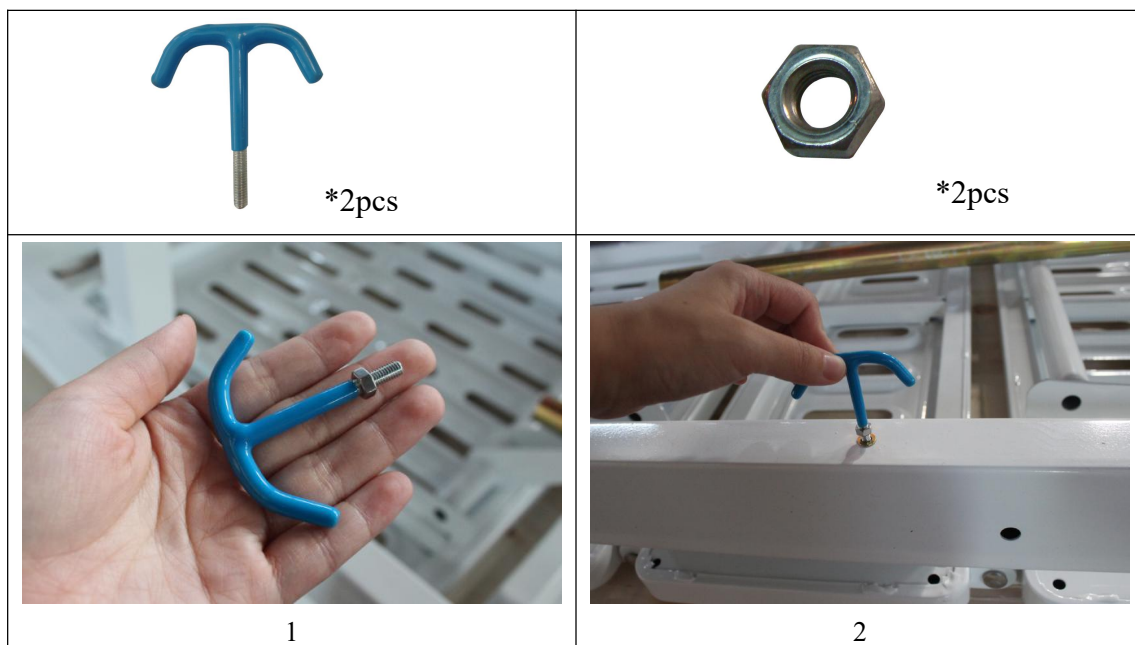

Step 3: Install the bed legs screws. Figure 3 shows (1-3) :

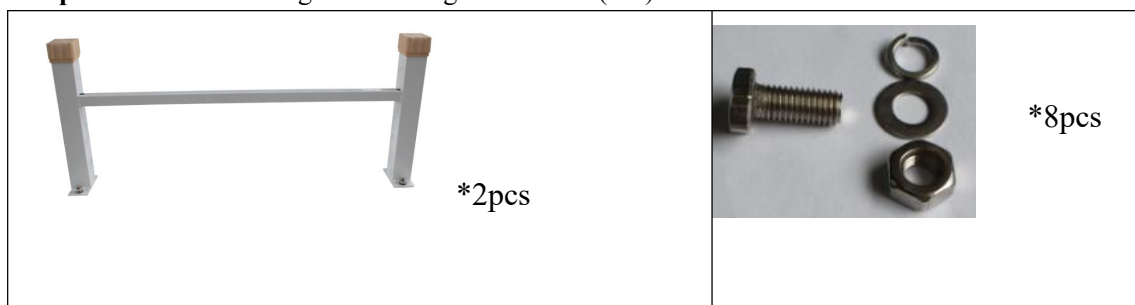

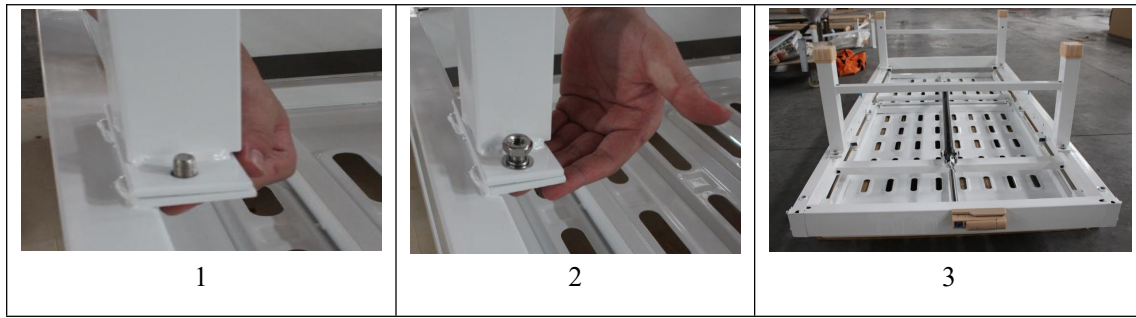


Step 4: Install the end of the bed headboard. Figure 6 shows (1-3) :

Two hooks at both side. Hold the head board to hook it on two steel column. important :
Cover the small steel plate down to lock the board.



6. Use Method

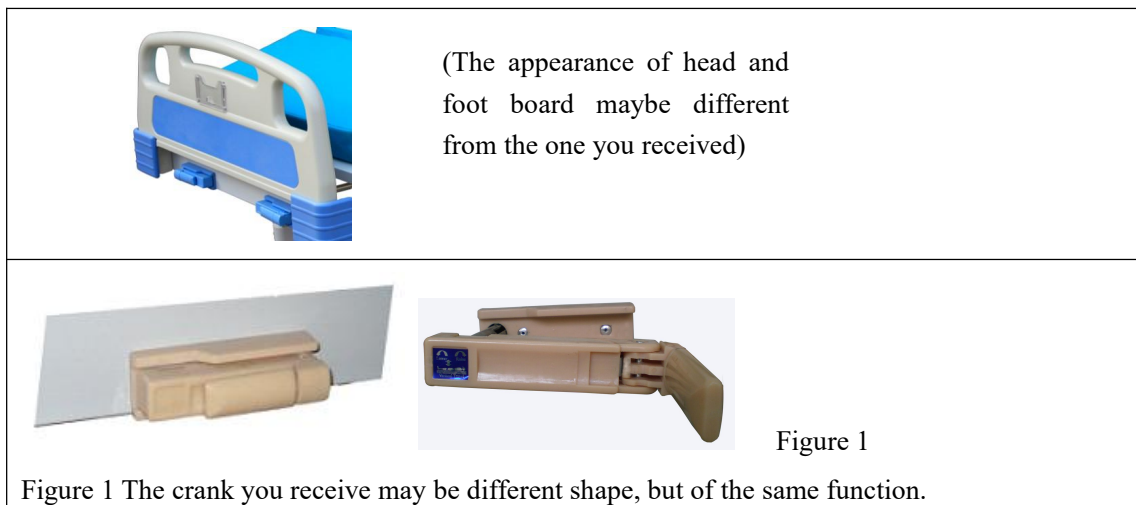

6.1 Function

6.1.1 Back-rest angle adjustable:



Swing the crank handle of "Back Rest" clockwise, to raise the back section.

Swing the crank handle of "Back Rest" anti-clockwise, to lower the back section(Figure 2).

6.1.2 Foot- rest angle adjustable



Swing the crank handle of "Foot Rest" clockwise, to raise the knee section

6.2 Using method of I.V pole:

Erect the I.V. pole when using. When the I.V. pole is upright, the height can be adjusted. When the height is not enough, rotate point of C and pull the pole B to be higher, and then clockwise point of C as locked, and put the infusion bag hang on the point A. (Figure 3)



Figure 3

6.3 Using method of casters:

If your bed is equipped with casters, you can move the beds with/without patient on bed.

Note: Lock the casters when you stop or locate the bed.

Brake pedals on casters. (Figure 4)

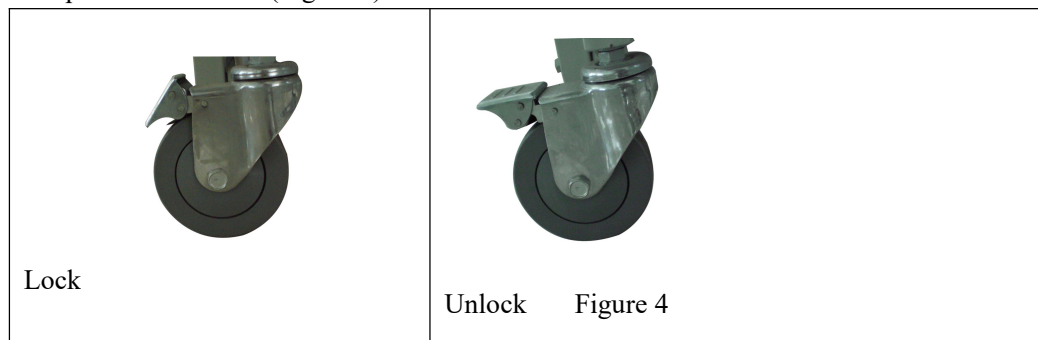


Figure 4

6.4 Drainage hook

Drainage hook is applied to the bed. It is used for hanging the urine bag. Install the drainage hook by twisting it to the bed. (Figure 5) If you do not need it, you can uninstall it.



Figure 5

7. Notes

7.1 Operators must read this manual carefully, follow the instructions and familiarize with the correct operating and maintenance procedures of the hospital bed.

7.2 When transferring the patients, do make sure to lock the guardrail. The company does not accept responsibility for any consequences resulting from incorrect operation.

7.3 Illegal and barbaric action and overload operation are prohibited, to avoid improper use damaging the hospital bed.

8. Warning and Caution

-  DO NOT ALLOW CHILDREN ON OR NEAR BED DURING OPERATING.
-  DO NOT ALLOW THIS DEVICE TO BE OPERATED BY CHILDREN.
-  WHEN OPERATING THE HI-LOW, KNEE, BACK FUNCTION OF THE BED, ALWAYS ENSURE THAT THE INDIVIDUAL CONFINED TO THE BED IS POSITIONED PROPERLY WITHIN THE CONFINES OF THE BED. DO NOT LET ANY EXTREMITIES PROTRUDE OVER THE SIDE OR BETWEEN THE BED RAILS WHEN PERFORMING ANY FUNCTIONS.
-  DO NOT USE UNAUTHORIZED SIDE RAILS.
-  WARNING/CAUTION LABELS APPLIED TO THE BED OUTLINE HAZARDS OR UNSAFE PRACTICES THAT COULD RESULT IN PERSONAL INJURY AND/OR PROPERTY DAMAGE.
-  WHEN USING NASAL OR MASKED TYPE ADMINISTERING EQUIPMENT, OXYGEN OR AIR TUBING MUST BE ROUTED AND SECURED PROPERLY TO ENSURE THAT TUBING DOES NOT BECOME ENTANGLED AND EVENTUALLY SEVERED DURING NORMAL OPERATION OF BED.
-  BEFORE OPERATING, INSURE THAT THE BED FRAME RIVETS ARE PROPERLY SEATED IN THE BED END HOOKS: OTHERWISE INJURY OR DAMAGE MAY RESULT.
-  KEEP ALL MOVING PARTS FREE OF OBSTRUCTIONS (I.E. BLANKETS/SHEETS, HEATING BLANKETS/PADS, TUBING, WIRING, AND OTHER TYPES OF PRODUCTS)

9. Maintenance

9.1 Keep clean as routine (Incl. disinfect).

9.2 Check if any parts loose or not as usual.

10. Storage


10.1 Store in the place of damp proof and non-corrosion environment

11. Transport

11.1 Be transferred by common transporting tools: ship, plane, train, truck.

11.2 Hospital bed can be manually handled.

11.3 No throwing or dropping during transshipment.

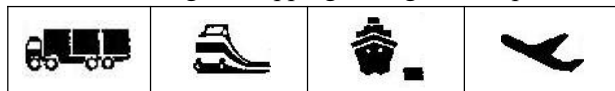

12. List of product

No.	Name	Quantity
1	Bed Frame	1set
2	Head and foot board	1set
3	Casters	4
4	I.V. pole	1
5	Side rail	1set
6	Drainage hooks	2
7	Mattress holder	1
8	Screws and nuts for installing	Some (refer to installing guide)
9	User's Manual	1
10	Nameplate	1
11	Lifting pole	1
12	Lifting pole handle	1



MODEL:SK033-1/ REF.:319-SK033-1

**CAMA HOSPITALARIA ARTICULADA PARA CUIDADOS
COMUNES/ARTICULATED HOSPITAL BED FOR COMMON CARE/ LIT
D'HOPITAL ARTICLE POUR LES SOINS COMMUNS/ LETTO OSPEDALIERO
PER DEGENZA A 4 SEZIONI / CAMA DE HOSPITAL ARTICULADA PARA O
CUIDADO COMUM/ VERSTELLBARES KRANKENHAUSBETT FUR
ALLGEMEINE PFLEGE**



Jiangsu Saikang Medical Equipment Co.,Ltd

No. 35 Lehong Road, Modern Agriculture
Demonstration Park, Zhangjiagang City,
Jiangsu Province, China



SUNGO Europe B.V.

Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel,
The Netherlands



QuirumedS.L.U.CIF B97267405

C/Corretger 117 Pq.Emp.Tactica,Paterna,
46980, Valencia(Spain)

LOT:HW25xxx



2025-xx-xx