



User manual

SKR-CT670-1 Color Painted Treatment Trolley



Picture for reference only

SAIKANG / Medical

Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098

Contents

Manual introduction	2
1. Installing guide	3
1.1 List of standard accessories	3
1.2 Casters installation	5
2. Use method	6
2.1 Use method of casters	7
2.2 Use method of buckets	8
3. Faults exclusion	9
4. Maintenance	9
5. Packaging, transport, storage	10
6. After-sales service	10

Manual introduction

Note:

Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.





- ▶ This manual is applicable to SKR-CT670-1 color painted treatment trolley. It contains the installation method, operating instructions and maintenance check points.
- ▶ The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- ▶ This user manual includes the description, instruction, maintenance and technical parameters of the product.
- ▶ Any questions, please contact the after-sale service center or distributor.
- ▶ Anyone using or operating this product should read and comply with it.
- ▶ Except for some unexpected situations, users are required to be familiar with the usage and precautions.

1. Installing guide

1.1 List of standard accessories


After opening the packing case, please check whether the following parts are complete in the packing case:

Table a:

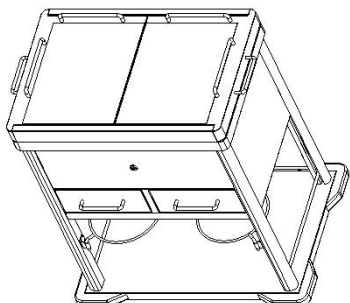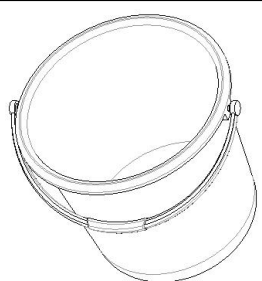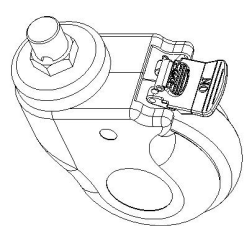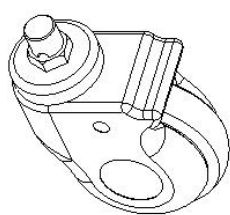NO.	Name	Picture	Specification	QTY	Unit
1	Main body		L670*W450*H920mm	1	pc
2	Bucket		φ233*205mm	2	pcs
3	Caster with brake		4inch caster with brake	2	pcs
4	Caster without brake		4inch caster without brake	2	pcs

4	Screw	/	Refer to Table a.1	1	set
---	-------	---	--------------------	---	-----





Table a.1

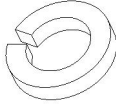NO.	Name	Picture	Specification	QTY	Unit
1	Spring washer		Φ12*Φ19.72*H3.54mm	4	pcs

1.2 Casters installation

Lay a protective pad and put the trolley body upside down;

Take out the $\Phi 12$ Spring washer from the screw package, and take out the castors, then insert them into the tube and tighten it clockwise with an inner hexagon wrench.

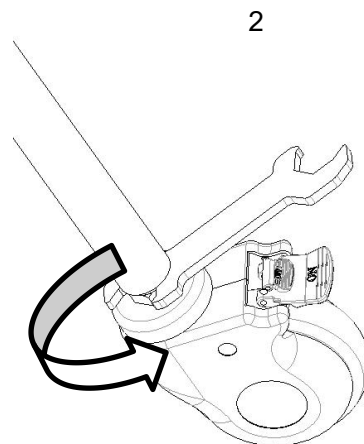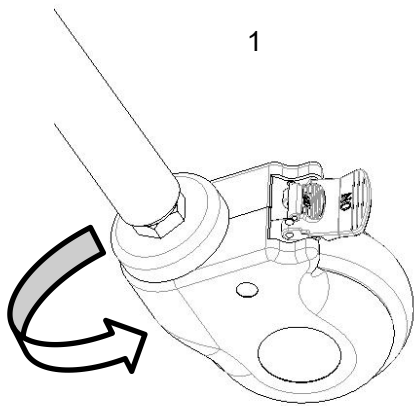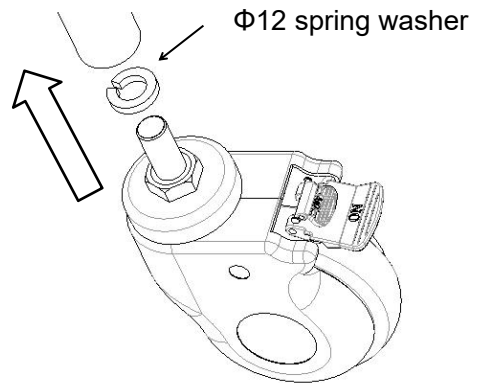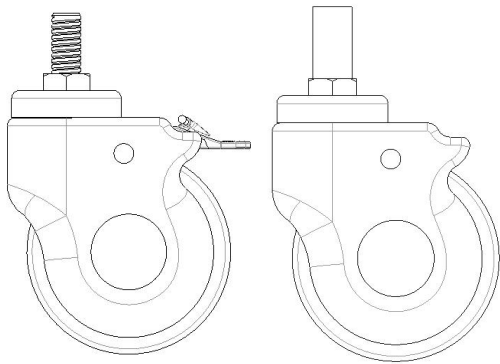

3

4

2. Use method




1-Color Painted Treatment Trolley 2-Castors


Trolley size	Length	mm	670
	Width		450
	The height of the trolley above the ground		950
Weight	Safe working load	Kg	50

2.1 Use method of casters

Lock the caster: After pushing the treatment trolley to the desired position, press the ON button with your foot and lock the caster when you hear a “click”, as shown in the figure below.

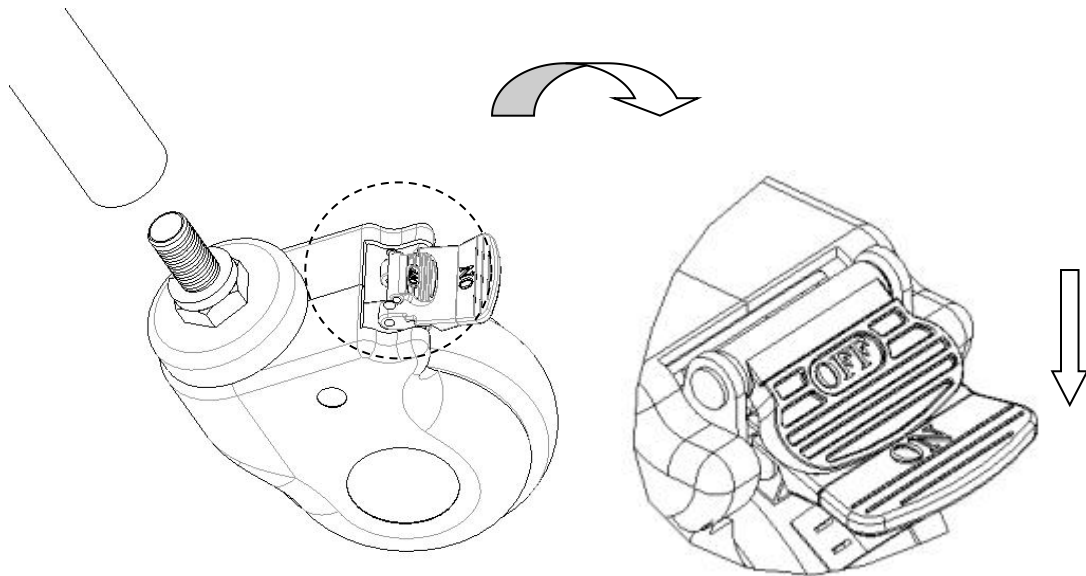


Unlock the caster: Press the OFF button with your foot and unlock the castor when you hear a “click”.

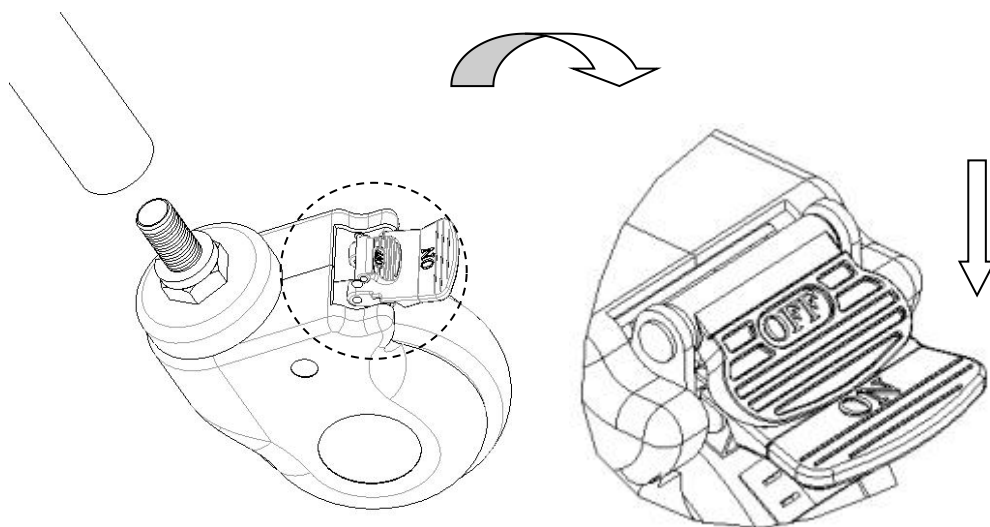

2.1 Use method of buckets

Turn the chine hoop and put the buckets in.

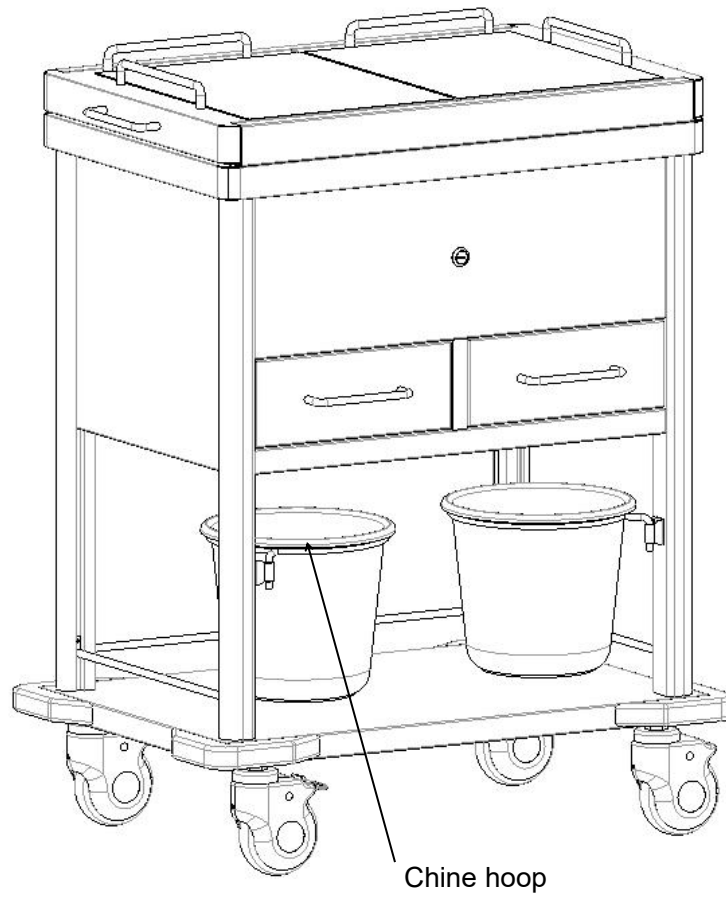

3.Faults exclusion

Common failures	Probable reasons	Excluding methods
Difficult to move the whole trolley	1. The caster is locked without releasing the brake 2. The caster is damaged	1. Release the brake,check 2.1 2. Replace the caster,check1.2

4. Maintenance

1.In order to use the treatment trolley safely, it is necessary to perform regular safety inspections on the treatment trolley. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.

2.When the treatment trolley frame ages and it reaches a certain servicing time, which the metal part of the treatment trolley and plastic parts of the treatment trolley frame can be recycle.

3.Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.

4.If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the treatment trolley, which will cause the stainless steel surface to rust.

5.Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.

6.It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.

7.The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.

8.Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.

9.For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a treatment trolley equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the treatment trolley also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels,

immediately replace the wheels and then retest the brake system.

10.If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

5. Packaging, Transport, Storage

1. The packaging of trolley is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the trolley.
3. The trolley should be stored:
 - a) Ambient temperature: $-40^{\circ}\text{C} \sim +70^{\circ}\text{C}$.
 - b) Relative humidity: $\leq 95\%$.
 - c) Atmospheric pressure: 500 hPa $\sim$ 1060 hPa.

6. After-sales service

1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification,the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD



MODEL:SKRCT670-1 | REF.: 319-SKRCT670-1

**CARRO HOSPITALARIO Y MÉDICO / MEDICAL TROLLEY / CHARIOT MÉDICAL / CARRELO
SANITARIO / CARRO MÉDICO / MEDIZINISCHER WAGEN**



Jiangsu Saikang Medical Equipment Co.,Ltd

No. 35 Lehong Road, Modern Agriculture
Demonstration Park, Zhangjiagang City,
Jiangsu Province, China



SUNGO Europe B.V.

Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel,
The Netherlands



Quirumed S.L.U. CIF B97267405

C/Corretger 117 Pq.Emp.Tactica,
46980, Paterna,Valencia(Spain)



LOT: HW25XXX



2025-xx-xx



Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098



No. 35, Lehong Road · Zhangjiagang (Jiangsu) · China

+86 18021231901

sales@chinasaikang.com

www.saikangmedical.com