



# User manual

## SKR-ET625 Emergency Trolley



**SAIKANG** / Medical

Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098

## 1.Casters installation

Align the wheel bolt with the mounting hole on the bottom of the cart and insert the wheel. Insert the screw into the mounting hole and tighten the screw clockwise.



1




2




3

## 2. Use method of casters

**Lock the caster:** After pushing the anesthesia trolley to the desired position, press the ON button with your foot and lock the caster when you hear a “click”, as shown in the figure below.

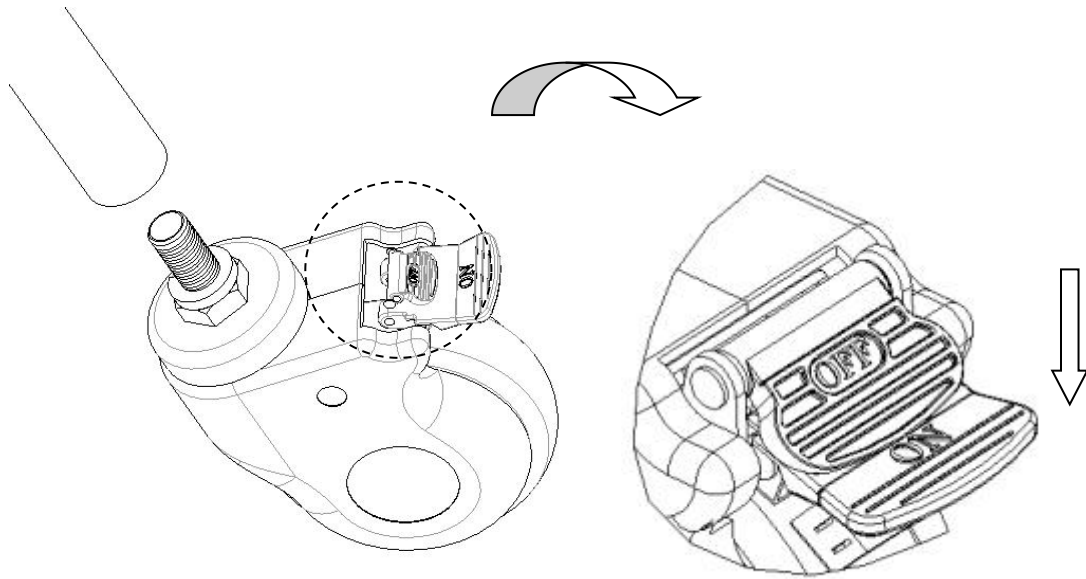

**Unlock the caster:** Press the OFF button with your foot and unlock the caster when you hear a “click”.

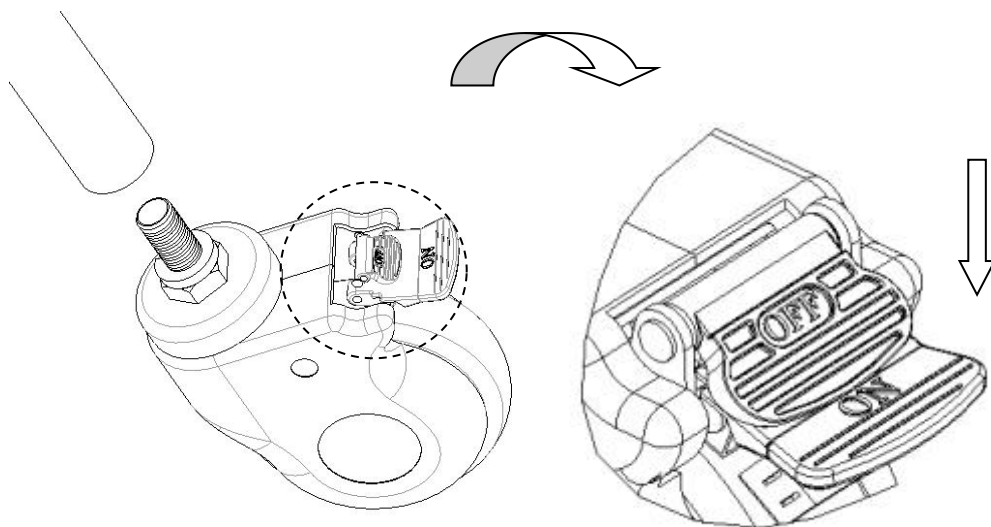

### 3. Faults exclusion

| Common failures                     | Probable reasons                                                                | Excluding methods                                                   |
|-------------------------------------|---------------------------------------------------------------------------------|---------------------------------------------------------------------|
| Difficult to move the whole trolley | 1. The caster is locked without releasing the brake<br>2. The caster is damaged | 1. Release the brake, check 2.1<br>2. Replace the caster, check 1.2 |

### 4. Maintenance

1. In order to use the anesthesia trolley safely, it is necessary to perform regular safety inspections on the anesthesia trolley. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.

2. When the anesthesia trolley frame ages and it reaches a certain servicing time, which the metal part of the anesthesia trolley and plastic parts of the anesthesia trolley frame can be recycled.

3. Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.

4. If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the anesthesia trolley, which will cause the stainless steel surface to rust.

5. Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.

6. It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.

7. The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.

8. Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.

9. For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For an anesthesia trolley equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the anesthesia trolley also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.

10.If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested.

## 5. Packaging, Transport, Storage

1. The packaging of trolley is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and prevent from sunlight and rain during the transportation of the trolley.
3. The trolley should be stored:
  - a) Ambient temperature:  $+5^{\circ}\text{C} \sim +40^{\circ}\text{C}$ .
  - b) Relative humidity:  $\leq 85\%$ .
  - c) Atmospheric pressure:  $70 \text{ kPa} \sim 106 \text{ kPa}$ .

## 6. After-sales service

1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification,the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD



**MODEL:SKR-ET625 | REF.: 319-SKR-ET625**

**CARRO HOSPITALARIO Y MÉDICO / MEDICAL TROLLEY / CHARIOT MÉDICAL /  
CARRELO SANITARIO / CARRO MÉDICO / MEDIZINISCHER WAGEN**



**Jiangsu Saikang Medical Equipment Co.,Ltd**

No. 35 Lehong Road, Modern Agriculture  
Demonstration Park, Zhangjiagang City,  
Jiangsu Province, China



**SUNGO Europe B.V.**

Fascinatio Boulevard 522, Unit 1.7,  
2909VA Capelle aan den IJssel,  
The Netherlands



**QuirumedS.L.U.** CIFB97267405  
C/Corretger 117 Pq.Emp.Tactica,Paterna,  
46980, Valencia(Spain)

**LOT:**

