



User manual

SKR002-4.640
ABS Trolley



SAIKANG / Medical

Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098

Contents

Manual introduction	2
1. Installing guide	Error! Marcador no definido.
2. Use method	6
2.1 Use method of casters.....	7
3. Faults exclusion.....	8
4. Instrument trolley-maintenance	8
5. Packaging, transport, storage.....	<u>9</u>
6. After-sales service	<u>9</u>

Manual introduction

Note:

Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.

- ▶ This manual is applicable to SKR002-4 instrument trolley. It contains the installation method, operating instructions and maintenance check points.
- ▶ The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- ▶ This user manual includes the description, instruction, maintenance and technical parameters of the product.
- ▶ Any questions, please contact the after-sale service center or distributor.
- ▶ Anyone using or operating this product should read and comply with it.
- ▶ Except for some unexpected situations, users are required to be familiar with the usage and precautions.

1. Installation guide

ABS Trolley

Picture



Technical parameters

Versatile multi-purpose cart suitable for use in multiple departments

Model: SKR002-4 Dimensions: 640 x 456 x 860 mm

1. The work surface is made of ABS material using injection molding technology in a single piece; features three tiers, one side with a handrail, and two drawers;
2. Equipped with 3-inch silent casters, two of which have braking functionality;
3. The cart can withstand a load of 30-50 kg and operates quietly without any unusual noise, making it ideal for hospitals to minimize disturbance to patients.



Packaging

Package dimensions: 690 x 305 x 485 mm

Package volume: 0.1 m³

Gross weight: 9.9 kg



Installation Guide

Warm reminder :

Please ensure that the trolley is installed correctly according to the manual.

Components



- | | |
|--------------------|------------------|
| ① Handle*1 | ⑤ Short cap*2 |
| ② Panel*1 | ⑥ Drawer*2 |
| ③ Vertical posts*8 | ⑦ Caster*4 |
| ④ Base plate*1 | ⑧ Middle panel*1 |

Installation



1. Install the back of the panel on the drawer, and press the small pillars on the panel with your hand until they can be inserted into the drawer.



2: Place the panel with the drawer in place.



3. Install the handle on the right side as shown in the picture, and place two short caps on the left side.



4. Flip the panel upside down and tighten the four vertical posts.



5. Install double-headed threaded connectors on top of the four vertical posts.



6. Cover the middle panel.



7. Install and tighten the four vertical posts.



8. Cover the bottom panel.



9. Secure the four casters. Two casters with brakes are recommended for diagonal installation.



10. Completed assembly diagram.

Application

Product applications: hospitals, medicine delivery, treatment, monitoring devices, electrocardiogram machines; beauty salons, nursing homes, welfare institutions, sanatoriums, hotels, pet stores, schools, 4S stores, etc.;

2. Use method




1-Main body 2-Casters


Instrument trolley size	Length	mm	640
	Width		456
	The height of the trolley above the ground		860
Weight	Safe Working load	KG	50

2.1 Use method of casters

2.1.1 Lock the caster: After pushing the trolley to the desired position, press the button and lock the caster when you hear a “click”, as shown in the figure below.

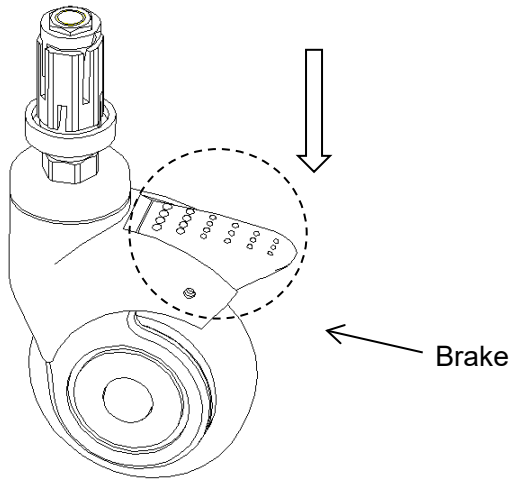

2.1.2 Unlock the caster: Lift the button with your foot and unlock the castor when you hear a “click”.

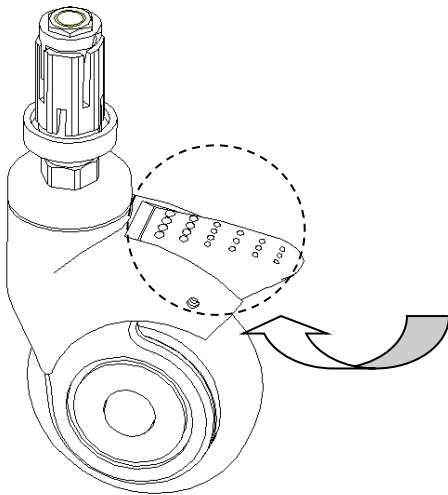

3. Faults exclusion

Common failures	Probable reasons	Excluding methods
Difficult to move the whole trolley	1. The caster is locked without releasing the brake 2. The caster is damaged	1. Release the brake 2. Replace the caster

4. Instrument trolley-maintenance

1. In order to use the trolley safely, safety inspections must be done regularly. Conduct a comprehensive inspection every six months, to ensure that there is no looseness in the connection parts can be operated normally.

2. When the instrument trolley reaches its maximum service life, the metal and plastic parts of the crib can be recycle.

3. Avoid hitting the crib and scratching the tube on the crib with sharp objects.

4. If the structure with stains, we suggest to clean it with water, don't use alkaline,corrosive chemicals to clean, avoiding rust on stainless steel surface.

5. Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.

6. It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.

7. The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.

8. Judging whether the caster is replaced by the whole caster according to the wear and tear. After the wheel is replaced or installed, make sure that the axle bolt and nut are connected reliably. If possible, use a lock washer or lock nut.

9. For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a instrument trolley equipped with multiple brake casters, only one brake caster can be locked at a time to try to push the trolley to check whether the brake performance of each caster is good. If the function of the caster brakes fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.

10.If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales service department or an authorized dealer authorized by our company. Every time the brakes are replaced, the brake performance of the casters needs to be retested.

5.Packaging, transport, storage

1. The packaging of instrument trolley is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and protection from sunlight and rain during the transportation of the instrument trolley.
3. The instrument trolley should be stored:
 - a) Ambient temperature: -40°C ~ 70°C.
 - b) Relative humidity: ≤ 95%.
 - c) Atmospheric pressure: 500hPa ~ 1060 hPa.

6.After-sales service

1. Please keep the files come with the machine and invoices of this product properly, and you need to present these materials when the company performs warranty and maintenance for the product.
2. If there is any problems in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. Do not disassemble the internal device of this product by yourself to avoid unnecessary damage. If you find that the use is affected due to quality problems, please contact our after-sales service department or our authorized dealer.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will repair the product for users free of charge.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD



MODEL:SKR002-4.640 | REF.: 319-SKR002-4.640

**CARRO HOSPITALARIO Y MÉDICO / MEDICAL TROLLEY / CHARIOT MÉDICAL / CARRELO SANITARIO /
CARRO MÉDICO / MEDIZINISCHER WAGEN**



Jiangsu Saikang Medical Equipment Co.,Ltd

No. 35 Lehong Road, Modern Agriculture
Demonstration Park, Zhangjiagang City,
Jiangsu Province, China



SUNGO Europe B.V.

Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel,
The Netherlands



Quirumed S.L.U. CIF B97267405

C/Corretger 117 Pq.Emp.Tactica,Paterna,
46980, Valencia(Spain)

LOT:HW25036



2025-xx-xx



Jiangsu Saikang Medical Equipment Co., Ltd.

STOCK CODE: 870098



No. 35, Lehong Road · Zhangjiagang (Jiangsu) · China

+86 18021231901

sales@chinasaikang.com

www.saikangmedical.com