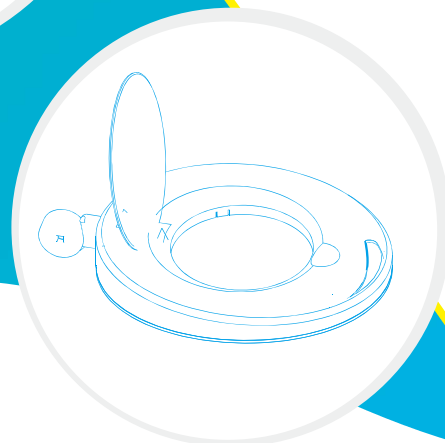
USER MANUAL

LED Medical Lights

ESPAÑOL-----Lámparas médicas LED serie KS
ENGLISH -----KS Series LED Medical lights
PORTUGUÊS-----Luzes médicas LED série KS
ITALIANO----- Lampade mediche a LED serie KS
DEUTSCH----- Medizinische LED-Leuchten der KS-Serie
FRANÇAIS----- Lampes médicales à LED série KS

IMPORTER : QUIRUMED S.L

CALLE CORRETER 117 PATERNA 46980 ESPAÑA

CE MADE IN PRC

Potencia/Power: AC 220V~22V 50Hz±1Hz

LOT: VER CAJA/SEE OUTER BOX

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Model: KS Sereis LED Medical Lights

Heat Management

LED: IR-free technology from LED light source reduces the sensation of heat for the patient and doctor. Tissue do not dry up as rapidly during the intervention under very low radiation from the light

Positioning and long reach

articulated arm ensures the light is easy to move and position and dose not drift. NO dead-angle examination and long reach range is presented.

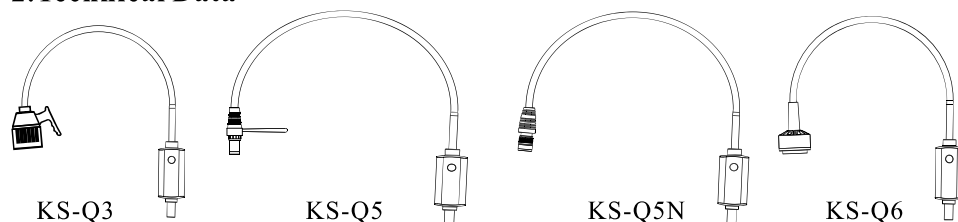
Energy-Saving

LED service life 40 times longer than Halogen bulb. Low energy consumption : 1/3 less than conventional Halogensystem


Application Range

Close examination ,Over bed, Intensive care, Recovery rooms, Dermatology,Aged care, Gynaecology, Ophthalmology, E.N.T , minor surgical, Home Decoration, office and work station etc.

2.Technical Data


Model	KS-Q3	KS-Q5	KS-Q5N	KS-Q6
Color	Black/ White	Black	Black	Black/ White
Light source	1xLED bulb,3W	1xLED bulb,5 W	1xLED bulb,5W	1xLED bulb,6 W
Diameter of lamphead	75mm	25mm	35mm	80mm
Light field diameter	80-150mm (30-50cm)	70-150mm (30cm) Variable	50-200mm(30cm) Variable	70-200mm(30-100cm)
Illumination	70,000 Lux (30cm) 40,000 Lux (50cm)	18,000 Lux (30cm) 7,000 Lux (50cm)	50,000 Lux (30cm) 12,000 Lux (50cm)	78,000 Lux(30cm) 40,000 Lux (50cm) 20,000 Lux (100cm)
Flexible arm	760mm	1200mm	1200mm	760mm
Brightness control	Dimmer/ Option:lighttouch, touchless, battery			
Mounted type	Mobile / Option:Wall mounted, Rail clamp, table clamp			







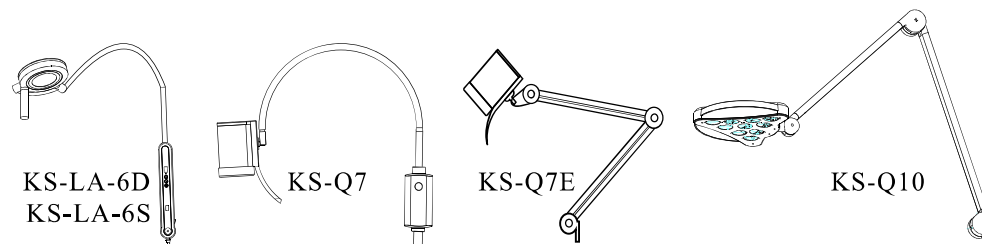
Model	KS-LA-6D /6S	KS-Q7	KS-Q7E	KS-Q10
Color	White	White/Orange	White/Orange	White
Light source	6xLED bulb,3W	7xLED bulb,10W	7xLED bulb,10W	10xLED bulb,6 W
Diameter of lamphead	120mm	89mm	89mm	270mm
Light field diameter	180-300mm (50-100cm)	120-200mm(50-100cm)	120-200mm(50-100cm)	100-200mm(50-100cm)
Illumination	45,000 Lux(50cm) 18,000 Lux(100cm)	78,000 Lux (30cm) 60,000 Lux (50cm)	78,000 Lux (30cm) 60,000 Lux (50cm)	78,000 Lux (50cm) 40,000 Lux (100cm)
Flexible arm	760mm	760mm	94mm spring arm	1315mm spring arm
Brightness control	lighttouch Option: touchless, battery	lighttouch Option:batterybox		
Mounted type	Mobile / Option:Wall mounted, Rail clamp, table clamp,Ceiling mounted(LA-6,Q10)			

3. Interface of product

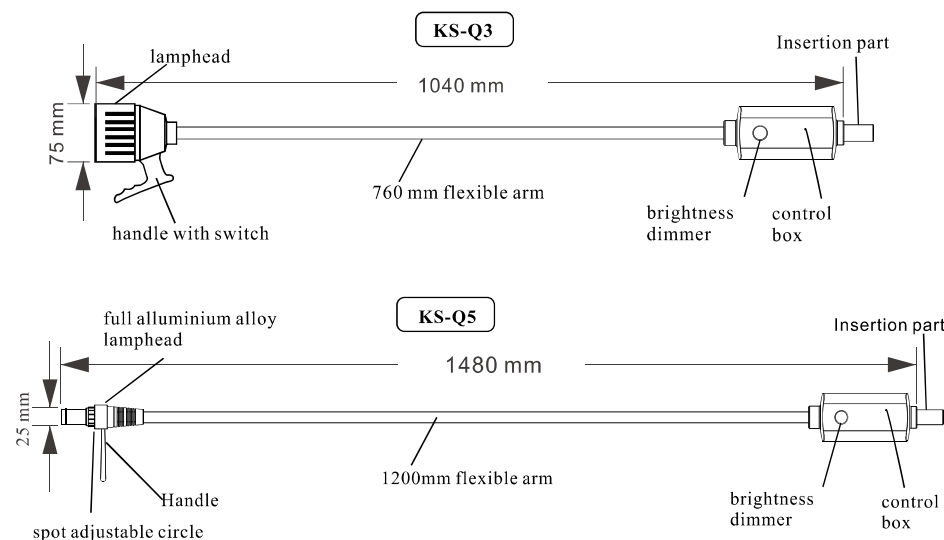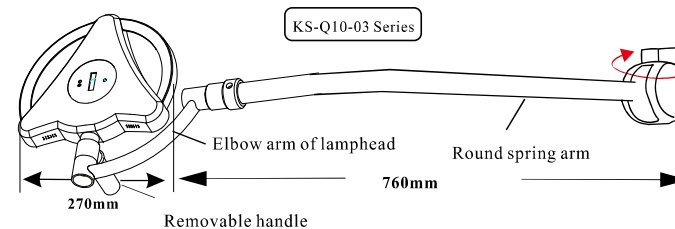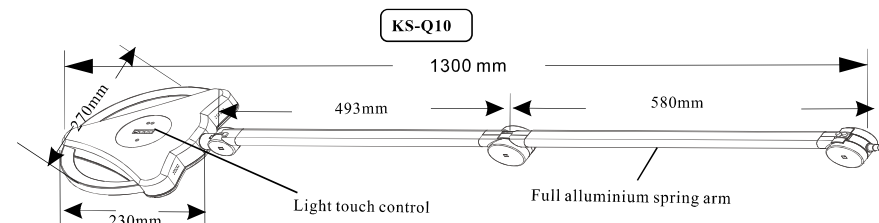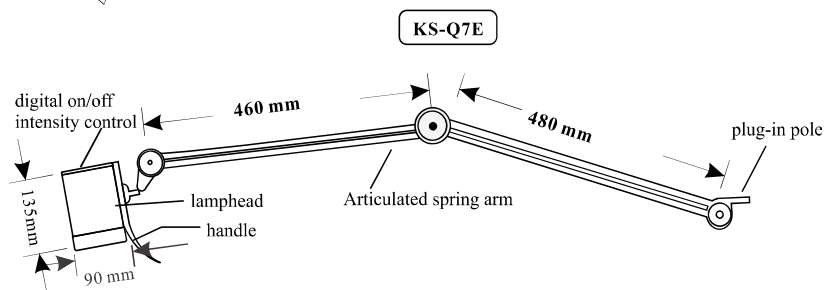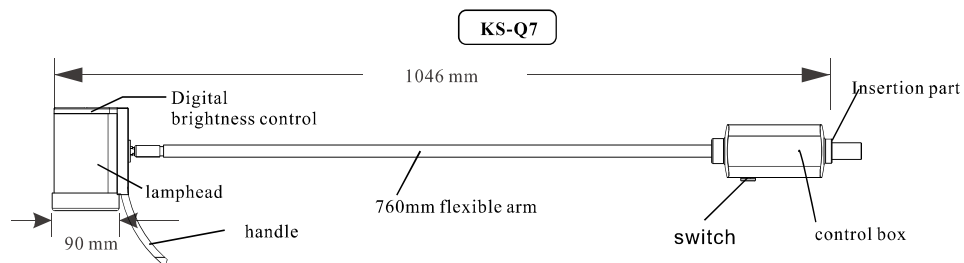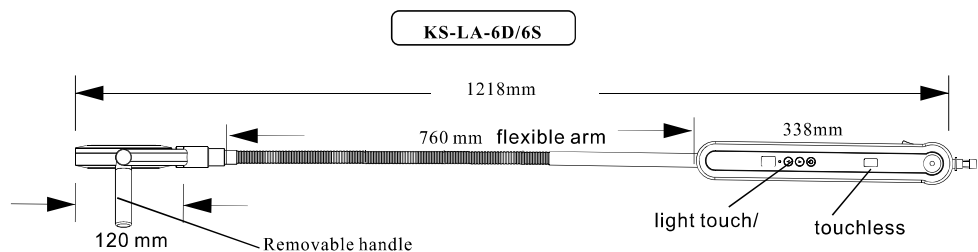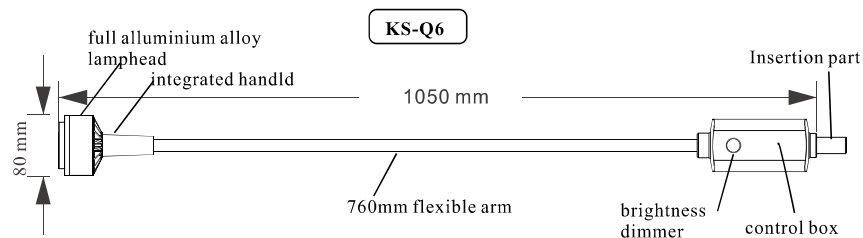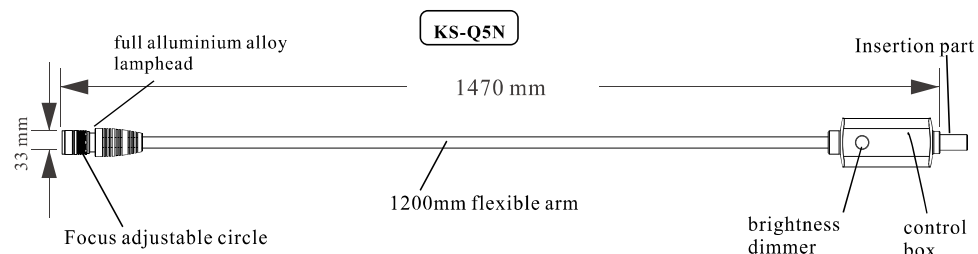A. Top part Including lamp head, flexible arm, control box

B. Mounted Type of lamps

B.1. Standard mobile for KS-Q3

	Code of base	ABS01 Black/ ABS02 White
	Material	Integrated plastic
	Castors	D=50mm with 5 braker
	Base	D=530mm, 5 branches, 1.5kgs
	Vertical pole	L=850mm



B.2. Standard mobile for KS-Q5, Q5N, KS-Q6

	Code of base	DM01 Black/ White
	Material	Thicker powder coated steel
	Castors	D=50mm with 5 braker
	Base	D=580mm, 5 branches, 3.4kgs
	Vertical pole	L=660mm/ 850mm


B.3. Standard mobile for KS-Q7/Q7E, KS-LA-6D/6S/6B, KS-Q10

	Code of base	DM02 Black/ White
	Material	Thicker powder coated steel
	Castors	D=60mm with 5 braker
	Base	D=580mm, 5 branches, 3.4kgs/ 6kgs(DM03)
	Vertical pole	L=850mm

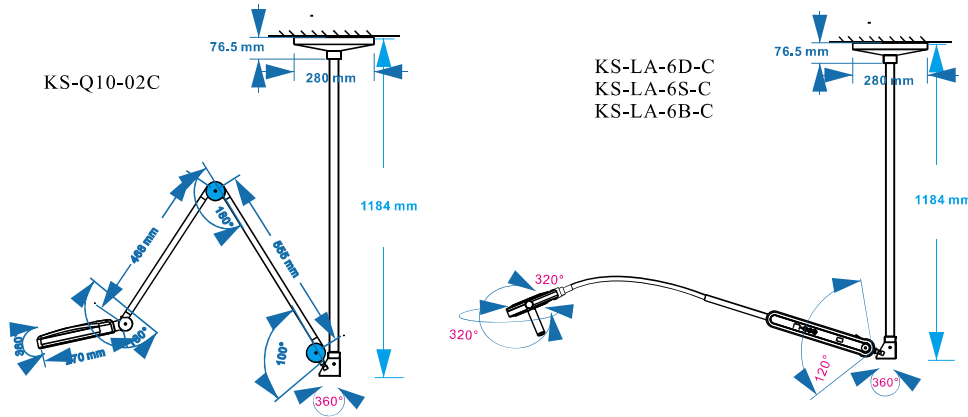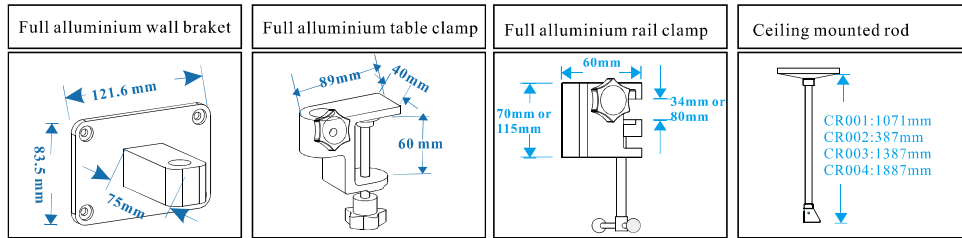B.4. Optional mounted parts

B.5. KS-Q10 03 Series mounted parts

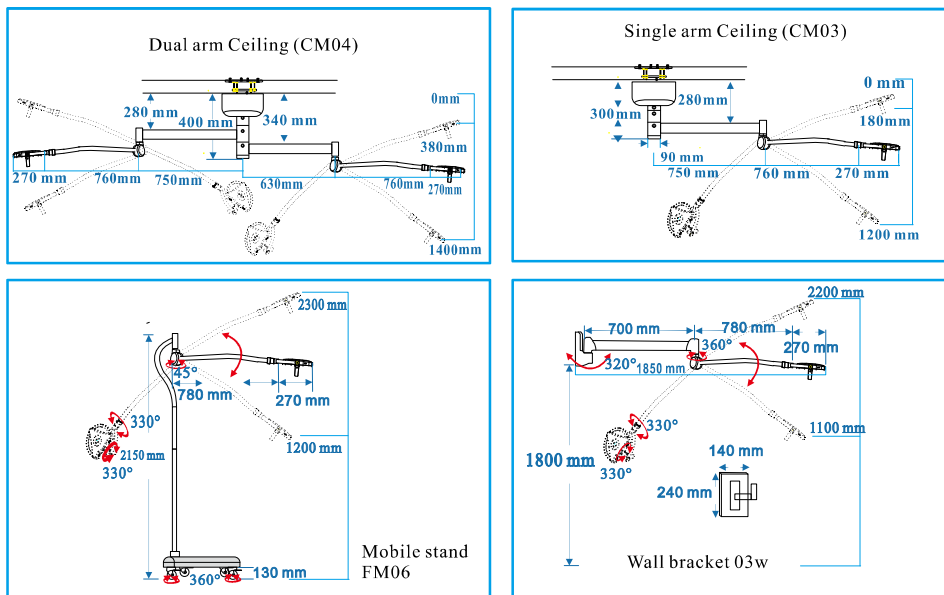4.Installation and adjustments

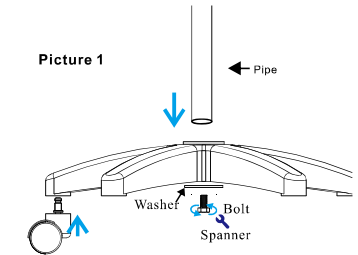
4.1. Check the main body and get through power supply

Open the package, and check if everything is ok. If any problem, pls contact manufacturer or distributors. Take out the lamp from package, and connect to the power supply, turn on the light.

4.2. Mobile installation


A. Plastic mobile base:

Insert the 5 plastic castors into the five branches base. Take out the vertical metal pipe, insert into the center hole of the plastic base, lock the longer screw from the center of the bottom (Refer to picture 1)

B. Metal detachable base:

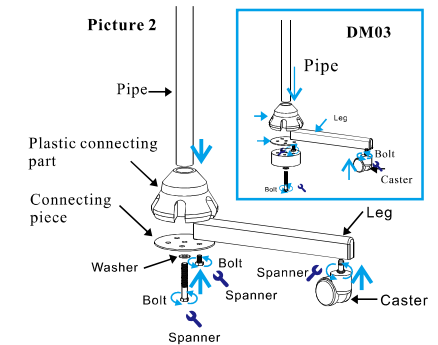Take out the legs and screw the castors. Second, put legs into the slots of the plastic connecting part, put the metal connecting piece from the bottom and screw the bolts to fix the legs in the connecting part. Finally, insert the vertical metal pipe into the plastic connecting part, (if for DM03, put the additional iron block from bottom base) and lock the longer screw from the center of the bottom. (Refer to picture 2)

4.3. Wall mounted: put the setscrew in the wall and screw the clipper on the wall by mating holes.

4.4. Table clamp: put this part to the table from the table edge, screw tightly from the bottom.

4.5. Rail clamp: unscrew the rod, make the upper and lower slipping block to fix the rail stick, screw tightly from the bottom.

4.6.Ceiling installation

Turn down the fixed ring of the top cover, locate it according to the installation hole of the top screw, install the expansion screw, and install the top according to the location of the expansion screw.

There is a high-voltage power supply line outlet (Attention: AC:110-240V) on the top. After the top is installed, the power supply line is connected with AC electric supply from Ceiling, and the fixing ring of the top cover is screwed back.

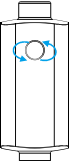
After the power cord is connected, the top part is installed, the upper part of the lamp is taken out, inserted into the installation hole for the lamp arm, and then inserted into the DC plug beside the installation hole, and the installation is completed.

4.7. Insert the main top body into the top of vertical metal pipe, screwing tightly from the side, connect the plug to electric socket and turn on for working.


5. Operation and function

5.1. Position adjustment of lamp head: Hold lamp head in one hand, hold vertical column assembly in one hand, pull lamp head to adjust lamp head position.

5.2 Brightness adjustment

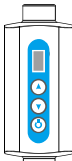
KS-Q35/Q3/Q5/Q5N/Q6:
Rotate the knob on the control box to dim the light brightness steplessly


KS-Q3D/Q5-3D/Q5-3B/Q5N-3D/Q5N-3B/Q6D/Q6B: light touch ▲ on the control box to increase or ▼ decrease the brightness adjustment range is 1-5 grade display on the LED screen.

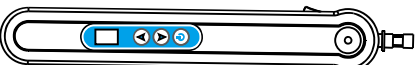KS-Q3S/Q5-3S/Q5N-3S/Q6S: The hand sweeps uniformly at a distance of 3-5cm from the sensor port, which can switch on/off and adjust brightness.

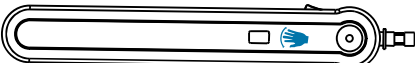
KS-LA-6D/6B: light touch ▲ on the control box to increase or ▼ decrease the brightness adjustment range is 1-5 grade display on the LED screen.

KS-LA-6S: The hand sweeps uniformly at a distance of 3-5cm from the sensor port, which can switch on/off and adjust brightness.

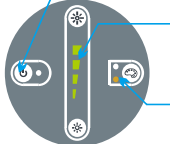KS-Q7/Q7E: Digital keypad ▲ on the lamphead to increase or ▼ decrease the brightness

KS-Q10
Power on/off
Light intensity
R9, Color temperature Adjusted

6. Working Condition

Environment Temperature : 5°C ~ +40°C

Relative Humidity : ≤85%

Atmospheric Pressure : 700hPa ~ 1060hPa

7. Notice

1. Before cleaning, take off the electric core firstly.
2. Liquid detergent is not allowed to clean.
3. Installation should be obey the instruction.
4. Before replacing the burnt bulb, the power supply of the inspection lamp should be cut off.
5. Installation, trouble shooting, adjustment should be operated by professional personnel.
6. Keep nothing placing on the electric core

8. Warranty and After-sale service

8.1. Warranty of 12 months from loading port

- The warranty period in normally using, the trouble because of the product quality, the consumer depends on the warranty slip and invoice enjoys free maintenance service.
- Without our factory permit after installation of other features, the fault doesn't lie within the scope of warranty.
- Mark on products such as model NO., date of the factory has been changed ,delete, move or not identify, will not enjoy free maintenance service.
- As falling ,squeezing, soak, damp and other human factors caused the fault is not within the scope of warranty.

8.2. After sales service

Customers are required to provide the products codes, image or video which will help us to confirm the problem. After acknowledging the problem, if it is easy to fix, we will send out the replacing part to customer for solving the problem. If problem is unrepairable, then will consider to return or resend.

9. Troubleshooting

Problem	Analysis	Resolution
The switch is on, but the light is not on	Adaptor is connected not well	Check the adaptor
	no power supply	Check the electric cord
	PCB defective	Check the AC power supply source
Brightness can not be adjusted	PCB defective	Contact manufacturer
		Contact manufacturer

10. Symbols

	Warning		Check attached files		Refer to the manual
	CE mark		European Representative		Medical Devices (MDR)
	Information of manufacturer		Date of manufacture		Product Code
	Lot number		Series number		WEE disposal
	Keep up		Handle with care		Away from sunlight
	Keep in dry		Recycling		Low-density polyethylene

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