Well-being made to measure

WELLCORE is born from the fundamental pillars of life: health and well-being. With the goal of making you feel good, WELLCORE offers you the peace of mind and confidence of knowing that you take care of yourself as much as you deserve in your daily life.

WELLCORE is a new brand offered exclusively by the Bunzl Group. As a company with extensive experience in the market and in health-related areas, Bunzl understands the need to bring personal care and well-being to people in a simple and affordable way.

Our pace of life does not stop and our well-being is constantly at stake, so it is vital to have products that guarantee a good balance. In our brand you will find a wide range of equipment to take care of yourself in a quick and easy way.

We are always working with new innovations and the latest technologies. You will discover modern and intuitive equipment that you can incorporate into every moment of your day to achieve that extra well-being you were looking for.

You don't have to worry, WELLCORE has arrived to simplify your idea of taking care of yourself. It's time to take care of yourself!

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MODELO/MODEL	REFERENCIA/REFERENCE	COLOR/COLOUR
WMD82-BK	WLLC1144-BK	BLACK
WMD82-LB	WLLC1144-LB	LIGHT BLUE
WMD82-PK	WLLC1144-PK	PINK
WMD82-SK	WLLC1144-SK	SKIN

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EN

DESCRIPTION: Cotton elastic bandage with acrylic adhesive. It gives traction on the skin or muscle to which it is applied, thanks to the hot fusion with the ability to recoil longitudinally, after being applied.

PURPOSE: Treatment of ailments, swelling and tensions caused by sports muscle injuries. Promotes blood circulation. Product under the supervision of a physician or physiotherapist.

HOW TO USE: Apply to clean, dry skin without wounds. Cut the elastic bandage to the desired length. Apply the bandage by spreading it according to the area to be treated and the desired effect. With due care to keep the underlying skin surface smooth and extended. The bandage can even be used for several days in a row. It is recommended to use the bandage under the advice of a doctor or physiotherapist.

PRECAUTIONS - STORAGE:

- Single-use product. Do not reuse.
- Avoid excessive tensions. If you see a very strong compression sensation, it indicates that the bandage is too tight.
- Do not squeeze too tightly in circulation. Do not overstretch the tape. The adhesive has been dermatologically tested, but do not apply in case of atopic skin or skin hypersensitivity. Remove in case of allergic reaction to avoid irritation or itching.
- The product is not sterile. Do not use on open wounds.
- Keep away, out of reach of children.
- Preferably use before the date indicated on the packaging. Any adverse reactions and serious incidents related to the product should be reported to the manufacturer and the competent authority (AEMPS).
- Avoid high temperature & moisture.

SPECIFICATIONS:

Size (unstretched)	5 m in length and 5 cm in width
Size (stretched)	9 m in length and 5 cm in width
Size (roll)	7 cm diameter x 5 cm
Weight	85 g
Composition	95% Cotton, 5% Spandex. Hypoallergenic Acrylic Glue
Reuse	NO, single use
Shelf life	5 years unopened

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WUXI WEMADE HEALTHCARE PRODUCTS CO., LTD
SUNGO EUROPE B.V. FASCINATIO BOULEVARDS 522, UNIR 1,7,